


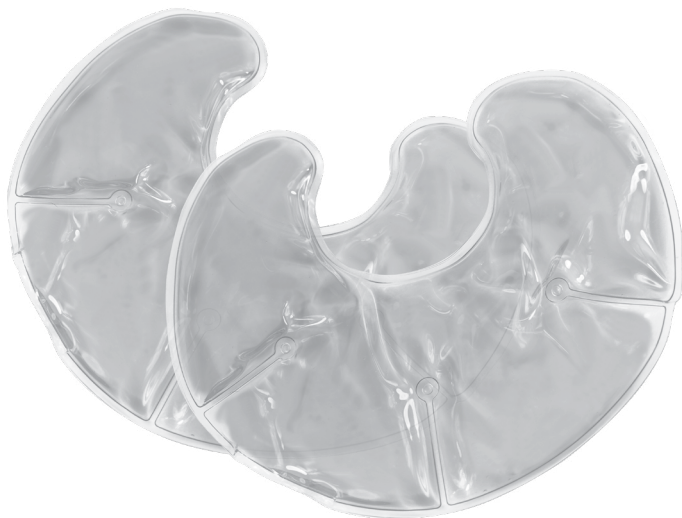

- IT 2 Cuscinetti in Gel Termico per il Seno
- ES 2 Almohadillas Térmicas para el Cuidado del Pecho
- EN 2 Thermal Gel Breast Pads
- FR 2 Coussinets d'Allaitement en Gel Thermique
- DE 2 Thermogel-Stilleinlagen



IT Manuale di Istruzioni

FR Manuel d'Instructions





ES Manual de Instrucciones

DE Bedienungsanleitung

EN Instruction Manual





A

IT USO A CALDO PER STIMOLARE IL FLUSSO DI LATTE PRIMA DELL'ALLATTAMENTO
ES USO CALIENTE PARA ESTIMULAR EL FLUJO DE LECHE ANTES DE DAR DE MAMAR
EN WARM USE TO STIMULATE THE MILK FLOW BEFORE NURSING
FR UTILISATION CHAUDE POUR STIMULER L'AFFLUX DE LAIT AVANT D'ALLAITER
DE GEBRAUCH VON WÄRME STIMULIERT DEN MILCHFLUSS VOR DEM STILLEN
PT USO QUENTE PARA ESTIMULAR O FLUXO DE LEITE ANTES DA AMAMENTAÇÃO
RO UTILIZARE LA CALD PENTRU A STIMULA FLUXUL DE LAPTE ÎNAINTE DE ALĂPTARE
SL TOPLA UPORABA ZA STIMULACIJO TOKA MLEKA PRED DOJENJEM
GR ΖΕΣΤΗ ΧΡΗΣΗ ΓΙΑ ΝΑ ΥΠΟΚΙΝΗΣΕΤΕ ΤΗ ΡΟΗ ΤΟΥ ΓΑΛΑΚΤΟΣ ΠΡΙΝ ΤΟΝ ΘΗΛΑΣΜΟ
SK TOPLA UPORABA ZA STIMULACIJO TOKA MLEKA PRED DOJENJEM
CZ TEPLÉ POUŽÍVÁNÍ K STIMULACI TOKU MLÉKA PŘED KOJENÍM
HU MELEGEN TÖRTÉNŐ HASZNÁLAT A TEJ ÁRAMLÁSÁNAK SERKENTÉSÉRE SZORZATÁS ELŐTT
BG ИЗПОЛЗВАТ СЕ ТОПЛИ, ЗА СТИМУЛИРАНЕ НА ПОТОКА ОТ МЛЯКО ПРЕДИ КЪРМЕНЕ

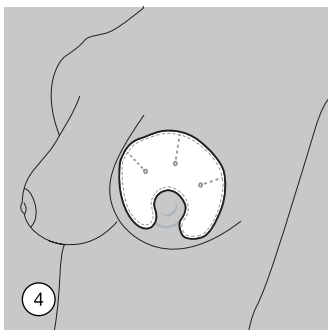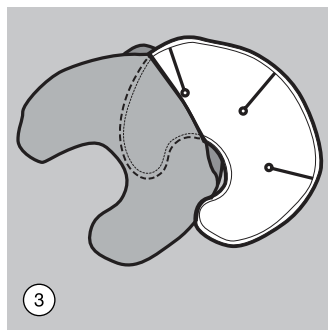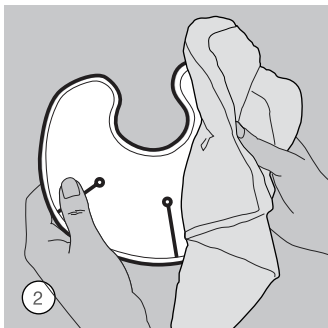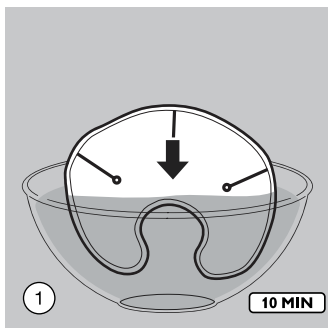

IT USO A FREDDO, O SUPER-FREDDO, PER ALLEVIARE IL SENO IRRITATO O GONFIO
ES USO FRÍO O EXTRAFRÍO PARA ALIVIAR EL PECHO IRRITADO O INFLAMADO
EN COLD AND EXTRA COLD USE TO RELIEVE IRRITATED OR SWOLLEN BREAST
FR UTILISATION FROIDE ET TRÈS FROIDE POUR SOULAGER LES SEINS IRRITÉS OU ENFLÉS
DE GEBRAUCH VON KÄLTE UND EXTRA KÄLTE BEI GEREIZTEN ODER GESCHWOLLENEN BRÜSTEN
PT USO FRIO E EXTRA FRIO PARA ALIVIAR O PEITO IRRITADO OU INCHADO
RO UTILIZARE LA RECE SAU FOARTE RECE PENTRU A CALMA IRRITAȚIILE SAU
SL HLADNA IN POSEBEJ HLADNA UPORABA ZA OMILITEV RAZDRAŽENIH ALI OTEKLIH PRSI
GR ΚΡΥΑ ΚΑΙ EXTRA ΚΡΥΑ ΧΡΗΣΗ ΓΙΑ ΝΑ ΑΝΑΚΟΥΦΙΣΕΤΕ ΤΟ ΕΠΕΘΙΣΜΕΝΟ Ή ΠΗΞΙΜΕΝΟ ΣΤΗΘΟΣ
SK HLADNA IN POSEBEJ HLADNA UPORABA ZA OMILITEV RAZDRAŽENIH ALI OTEKLIH PRSI
CZ STUDENÉ A VELMI STUDENÉ POUŽÍVÁNÍ PRO UVOLNĚNÍ PODRÁŽDĚNÍ NEBO NATEČENÍ PRSOU
HU HIDEG ÉS NAGYON HIDEG HASZNÁLAT AZ IRRITÁLÓDOTT VAGY DUZZADT MELLEK MEGNYUGTATÁSÁRA
BG ИЗПОЛЗВАТ СЕ СТУДЕНИ И МНОГО СТУДЕНИ ЗА ОБЛЕКЧАВАНЕ НА РАЗДРАЗНЕНИ ИЛИ ПОДУТИ ГЪРДИ

B

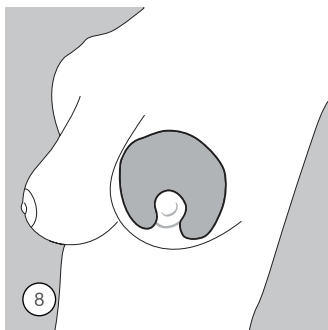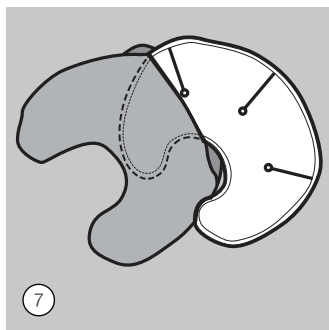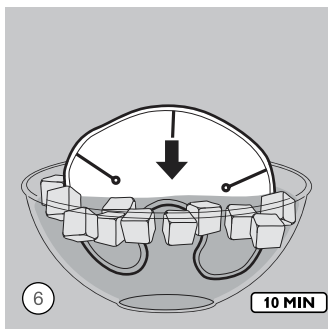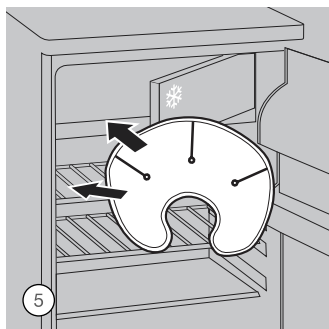

EN NUVITA 1256 - 2 Thermal Gel Breast Pads

Before using the product the first time, wash the sleeve and dry. It is mandatory to use the sleeve together with the thermopad.

Check the temperature of the thermopad through back of hand test before applying to your breast. Only apply the thermopad if the temperature has the correct warmth or coldness level.

(A) WARM USE TO STIMULATE THE MILK FLOW BEFORE NURSING

- 1) Soak the thermopad in a basin of warm water until it reaches an optimum and comfortable temperature.
- 2) Wipe to dry the thermopad.
- 3) Put the thermopad inside the sleeve.
- 4) Place the thermopad to the breast.

(B) COLD AND EXTRA COLD USE TO RELIEVE IRRITATED OR SWOLLEN BREAST

- 5) Place the thermopad in the refrigerator for a minimum of 1 hour or freezer for a minimum of 30 minutes.
- 6) You can also soak the thermopad in a basin of cold ice water
- 7) Wipe it dry and put the thermopad inside the sleeve
- 8) Place the thermopad to the breast

Cleaning and storing

Ensure to clean the thermopad with warm water and washing liquid prior to using the thermopad the first time and after every use. Never attempt to use any materials containing sharp edges or strong detergent. Place the thermopad in a cool and dry place away from children's reach.

Safety guidelines and diagnosis

- Do not use the thermopad with a microwave. Do not overheat.
- For external use only. Use the thermopad only with the sleeve included.
- Do not place thermopad directly on the skin when frozen.
- The thermopad must not be used while expressing milk or feeding your child. Do not apply to sensitive skin. Do not use on infants.
- Dispose the thermopad immediately with any immediate signs of damage.
- Do not fall asleep using the pack.
- Consult a physician for any inquiries about the use of this product.

Contents: 2 x Gel Thermopads, 2 x Sleeves

This product benefits of a warranty of 24 months on material and manufacturing defects, starting from the date of purchase

(see sales slip).

The warranty of 24 months does not include damages caused by the usual use of parties identified as "consumable" (e.g., batteries, brush heads or parts subject to usury).

The legal guarantee of 24 months is void if:

The product has undergone aesthetic damage due to improper use not in accordance with instructions in the manual.

This product has been modified and/or tampered with. The cause of the failure was due to poor maintenance of the individual components and/or accessories and/or supplies (e.g. oxidation and/or scaling due to the retention of water or other liquids, sediment blocking the sensor, leak of corrosive liquid from batteries).

The following is excluded from the legal guarantee of 24 months:

Costs related to replacements and/or repair of parts subject to wear or costs for ordinary maintenance of the product.

The costs and risks involved in transporting the product to and from the store where you purchased or otherwise authorized collection centre to receive the products under warranty.

Damage caused by or resulting from improper installation or improper use not in accordance with the directions in the instruction manual.

Damage due to natural disasters, accidental events or adverse conditions not compatible with the product.

Defects that have a negligible effect on product performance.

The manufacturer, distributor and all the parties involved in the sale do not assume any liability for losses and economic damage from any malfunction of the product. In accordance with current regulations the manufacturer, distributor and all the parties involved in the sale are not responding in any case for damages, including direct, indirect ones, loss of net income, loss of savings and additional damage and other details consequences going beyond the damage caused by the breach of warranty, contract, strict liability, wrongdoing or due to other causes, resulting from the use or inability to use the product and/or paper and electronic documents, including the lack of service.

For further information on the help service visit the website www.nuvitababy.com



info@nuvitababy.com

NUVITA® is a trademark distributed by Anteprema Brands International Ltd. - Europe
All rights reserved