nuvita

MATERNO GO

IT	Tiralatte elettrico
ES	Sacaleche eléctrico
EN	Electric breast pump
FR	Tire-Lait électrique
DE	Elektrische Milchpumpe
PT	Bomba tira-leite elétrica
GR	Ηλεκτρικό θήλαστρο
NO	Elektrisk brystpumpe
DA	Elektrisk brystpumpe
NL	Dutch: Elektrische borstkolf
ALB	Albanian: Pompë elektrike për gjirin
HU	Hungarian: Elektromos mellszívó

1282C

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Dear Customer, you bought one of the most reliable breast pumps available in the market. We studied this device to be as much user friendly as possible but, we are afraid, you still must spend a small amount of time in reading these instructions to get 100% of the benefits this product can bring you. Keep this manual for future reference.


PRECAUTIONS

- Please read the instruction manual carefully before using the product.
- This product must be used by adult only with complete attention during the entire process.
- Do not use the power adapter other than the provided.
- Do not warp or knot the connecting tube (Package Content No. 10) during the using process.
- Please put the product or dismantled parts out of baby's reach.
- Do not dismount Breast Pump Control Unit (Package Content No. 9) in any condition; it shall be repaired or replaced by the manufacturer or professional.

PACKAGE CONTENT

Before assembly and use the electric breast pump to make sure that inside the packaging there are the following accessories, refer to your shop in case of any missing parts:

1. Dust-proof cover
2. Silicone Massage cup
3. Breast Pump Body
4. Anti-spill silicone
5. Anti-spill cover
6. Suction Valve
7. Nuvita Baby Bottle (150ml)
8. Nuvita Bottle Silicone Valve
9. Breast Pump Control Unit
10. Connection Tube
11. Dust proof feeding bottle cover
12. Bottle Teat
13. Feeding bottle ring
14. USB - C Power Adapter
15. Velvet Bag (not shown in the diagram)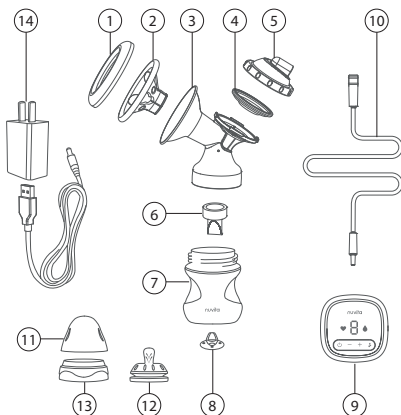

INSTRUCTION FOR CLEANING

- Before cleaning and sterilization, dismount the parts in contact with the breast or breast milk (apart from Breast Pump Control Unit (Package Content No. 9)).
- Do not clean the Breast Pump Control Unit (Package Content No. 9) in the water.
- Do not sterilize the Breast Pump Control Unit (Package Content No. 9) in the steam sterilizer.
- Do not hot sterilize the connection pipe. Cold sterilization only. Otherwise cleaning with warm soapy water and then rinse with sterile water. (Package Content No. 10).
- Clean the parts immediately after each use to avoid the residual milk and bacteria growth.
- Put the cleaned parts on the dry clean towel at the shady and cool place.

PRODUCT FEATURES

A rechargeable electric single breast pump model for maximizing milk extraction process. It's designed to simulate the suction rhythm of baby.

1. This Breast pump uses a 2 Phases Technology:
 - 1st Phase with 9 Automatic stimulation levels for 2 minutes operation.
 - 2nd Phase with 9 Selectable levels for your best Expression action
2. This device is provided with an anti-reflux protection system which prevents the breast milk to return to the control unit and damaging it during the milk extraction process.
3. Your 1282C is provided with memory function
4. Your 1282C is portable (AC/DC and Battery operated) so suitable for travelling and perfect for breast pumping activity at work.


DESCRIPTION OF PUMP CONTROL UNIT

- A. On and Off Switch button
- B. Mode Switch button
- C. Increase Suction Level button
- D. Decrease Suction Level button
- E. Level indicator
- F. Stimulation mode indicator
- G. Expression mode indicator




Frequency levels of absorption of Nuvita 1282C:

♥ 1 to 9 This is the Stimulation mode with fast and short suction rhythm.



💧 1 to 9 This is the Expression mode. This simulates the suction of a baby. The rhythm of this phase is slower, deeper and longer.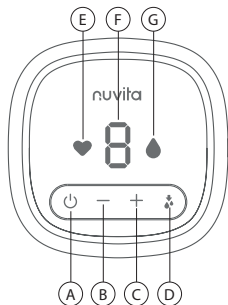

ASSEMBLY OF THE BREAST PUMP

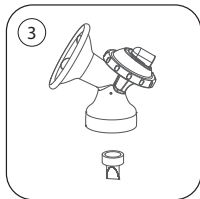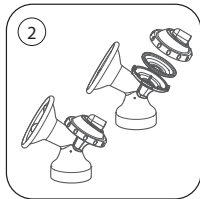
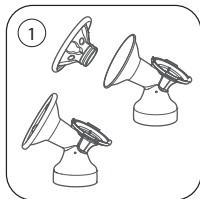
1. Insert the silicone massage cup (Package Content No. 2) in the breast pump as shown.
2. Fit the anti-spill silicon (Package Content No. 4) with the anti-spill cover (Package Content No. 5) as shown below. Aim at the sliding slot and screw the anti-spill cover as shown in the figure.
3. Fix the suction valve (Package Content No. 6) at the lower end of the breast pump see the figure below.
4. To connect the upper part of the breast pump with the wide neck bottle rotate until it fits close.
5. Insert the connection tube (Package Content No.10) into the interface of the anti-spill upper cover. Insert the other end of the connection tube into the suction slot of the pump.

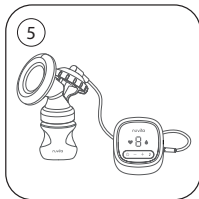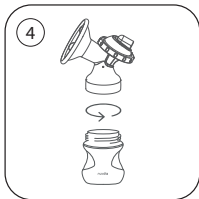USING THE BREAST PUMP

1. Please clean hands before using the product, ensure the parts of the breast pump are cleaned and assembled accurately as per instruction in this guide.
2. Relax and be at ease while having a normal sitting posture. Use a thermal gel breast pad or a hot towel before starting the pumping process.
3. Use your hand to aim the centre of the breast pump at the nipple and ensure it is tight and lock without air getting in and out.
4. Power on, long press the power button. The screen of the machine will self-test and flicker. The machine will then enter stimulation mode. The factory default level is level 5. After 2 mins of stimulation, the machine will automatically switch to expression mode. The machine will automatically shut off after 30mins working.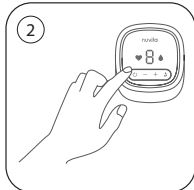
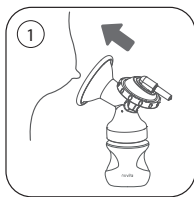


5. Under the Expression mode, press the + button or – button to select any levels from L1 to L9. This device is provided with Memory function. In the next breast feeding it will recall the previous level you selected.
6. Disassemble, clean and sterilize the non electronic parts for next use.

INSTALLATION OF AC ADAPTOR

IMPORTANT: Please turn off the machine before connecting with the adaptor.

1. Insert the USB-C port into the Control Unit and insert the AC power adapter into the power socket as shown in the diagram.
2. When battery is low capacity, the digit on the screen (level indicator) will flicker, indicating that battery needs to be charged.
3. When charging, the line of the digit on the screen will move clockwise and flicker, indicating that the machine is in charging mode as shown in the diagram. When machine full charged, it will auto shut off.

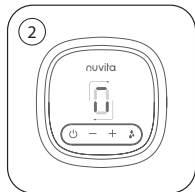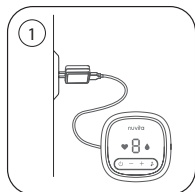WARRANTY - TERMS AND CONDITIONS

This product is covered by a 24-month warranty for manufacturing defects, starting from the date of purchase proven by receipt or invoice. In case of purchase with invoice, the legal warranty is reduced to 12 months. The 24-month warranty does not include damage resulting from normal wear and tear of the product, aesthetic damage, accessories or consumables (for example batteries, heads or components subject to wear). Batteries are guaranteed for a period of 6 months. Detailed warranty terms and conditions at

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Anteprima Brands International Ltd