# nuvitamed

- IT** Termometro digitale frontale / auricolare  
**ES** Termómetro digital para frente/oído  
**EN** Digital forehead / ear thermometer  
**FR** Thermomètre frontal/auriculaire numérique  
**DE** Digitale Stirn-/Ohrthermometer  
**PT** Termómetro digital de testa/ouvido  
**GR** Ψηφιακό θερμόμετρο μετώπου/αυτιού  
**NO** Digitalt panne-/øretermometer  
**DA** Digitalt pande-/øretermometer  
**NL** Digitale voorhoofd-/oorthermometer  
**ALB** Termometër dixhital ballor / për veshin  
**HU** Digitális homlok-/fülhőmérő  
**CZ** Digitální čelní / ušní teploměr  
**SI** Digitalni čelni / ušesni termometer  
**BG** Дигитален челен / ушен термометър  
**SK** Digitálny čelový / ušný teplomer  
**RO** Termometru digital frontal / auricular  
**HR** Digitalni čeonj / ušni toplomjer  
**LT** Skaitmeninis kaktos / ausies termometras  
**PL** Cyfrowy termometr czołowy / douszny  
**SB** Digitalni čeonj / ušni toplomer  
**GE** Digitales Stirn-/Ohrthermometer  
**SE** Digital pann-/örontermometer  
**FN** Digitaalinen otsa-/korvalämpömittari  
**EST** Digitaalne otsmiku-/kõrvatermomeeter  
**LV** Digitālais pieres / auss termometrs

2075

- IT** Manuale istruzioni d'uso  
**ES** Manual de instrucciones de uso  
**EN** Instruction manual  
**FR** Manuel d'instructions d'utilisation  
**DE** Bedienungsanleitung  
**PT** Manual de instruções de uso  
**GR** Εγχειρίδιο οδηγιών χρήσης  
**NO** Bruksanvisning  
**DA** Brugsanvisning  
**NL** Gebruiksaanwijzing  
**ALB** Manual përdorimi  
**HU** Használati útmutató  
**CZ** Návod k použití  
**SI** Navodila za uporabo  
**BG** Инструкция за употреба  
**SK** Návod na použitie

## INDEX

<b>IT</b>	Termometro digitale frontale / auricolare	003
<b>ES</b>	Termómetro digital para frente/oído	011
<b>EN</b>	Digital forehead / ear thermometer	019
<b>FR</b>	Thermomètre frontal/auriculaire numérique	027
<b>DE</b>	Digitales Stirn-/Ohrthermometer	035
<b>PT</b>	Termómetro digital de testa/ouvido	043
<b>GR</b>	Ψηφιακό θερμόμετρο μετώπου/αυτιού	051
<b>NO</b>	Digitalt panne-/øretermometer	059
<b>DA</b>	Digitalt pande-/øretermometer	067
<b>NL</b>	Digitale voorhoofd-/oorthermometer	075
<b>ALB</b>	Termometër dixhital ballor / për veshin	083
<b>HU</b>	Digitális homlok-/fülhőmérő	091
<b>CZ</b>	Digitální čelní / ušní teploměr	099
<b>SI</b>	Digitalni čelni / ušesni termometer	107
<b>BG</b>	Дигитален челен / ушен термометър	115
<b>SK</b>	Digitálny čelový / ušný teplomer	123
<b>RO</b>	Termometru digital frontal / auricular	131
<b>HR</b>	Digitalni čeoni / ušni toplomjer	139
<b>LT</b>	Skaitmeninis kaktos / ausies termometras	147
<b>PL</b>	Cyfrowy termometr czołowy / douszny	155
<b>SB</b>	Digitalni čeoni / ušni toplomer	163
<b>GE</b>	Digitales Stirn-/Ohrthermometer	171
<b>SE</b>	Digital pann-/örontermometer	179
<b>FN</b>	Digitaalinen otsa-/korvalämpömittari	187
<b>EST</b>	Digitaalne otsmiku-/kõrvatermomeeter	195
<b>LV</b>	Digitālais pieres / auss termometrs	203

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## 1. INTRODUCTION

This digital infrared thermometer allows for quick measurement of body temperature via the forehead or ear. It is suitable for adults and children (ear mode is only permitted for children over 3 months of age). Before use, read the manual carefully and keep it for future reference.

## 2. SAFETY WARNINGS

1. Keep the device out of reach of children under 12 years of age.
2. Do not immerse the thermometer in water or liquids. Clean only as indicated in the "Maintenance" section.
3. Use the product only for measuring body temperature.
4. Store between 5°C and 40°C, in a dry, well-ventilated place away from sunlight and humidity >95%.
5. Do not use or repair the thermometer if it is damaged. Contact your retailer.
6. The device contains precision components: avoid dropping, knocking or twisting the probe.
7. Even at normal temperature, symptoms such as vomiting, convulsions or lethargy require medical evaluation.
8. Patients taking antibiotics or antipyretics may not show fever. Do not rely solely on temperature.
9. Consult a doctor immediately in case of fever in people over 60 years of age, individuals with diabetes or who are immunocompromised (HIV, chemotherapy, steroids, etc.), bedridden patients or transplant recipients.
10. Not suitable for premature or low birth weight babies.
11. Do not allow children to use this product without supervision.
12. Do not use for continuous monitoring.
13. Avoid taking measurements during or immediately after breastfeeding.
14. Do not eat, drink or exercise before taking the measurement.

## 3. PRODUCT DESCRIPTION AND TEMPERATURE

### OPERATION

The thermometer detects infrared radiation emitted from the eardrum or forehead. The sensor converts the IR energy into a value shown on the LCD display.

### MAIN COMPONENTS

- Infrared sensor
- Body with multifunction button
- Backlit LCD display
- Microcontroller electronic circuit
- Buzzer and indicator lights
- Battery cover, probe cover

Measurement site	Normal range
Front	36.1°C - 37.5°C
Ear	35.8°C - 38°C
Mouth	35.5°C - 37.5°C
Rectum	36.6°C - 38°C
Axillary	34.7°C - 37.3°C

**NOTE:** Values from different sites are not directly comparable. Always inform your doctor of the site and type of thermometer used.

## 5. MAIN FEATURES

---


- Rapid measurement (<1 second)
- Front, ear and object modes
- Automatic forehead/ear recognition (models with magnetic probe cover)
- Memory for 40 measurements
- Fever alert with audible signal and colour backlighting
- Conversion °C/°F
- Automatic switch-off after 13 seconds
- Silent/active mode
- 4-colour backlight: white (object), green (normal), orange (mild fever), red (high fever)
- Distinctive technology to differentiate between adult and child measurements

## 6. DISPLAY

---

### ELEMENTS DISPLAYED ON THE SCREEN:

1. Adult treatment methods
2. Terms and conditions for children (under 12yrs of age)
3. Ear mode
4. Subject matter
5. Unit °C/°F
6. Last stored value
7. Low battery indicator
8. Audio on/off icon
9. Current temperature detected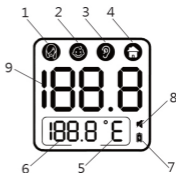

## 7. PREPARATION FOR MEASUREMENT

---


### TO AVOID MISTAKES:

- Use the device indoors, away from strong draughts.
- Avoid sudden movements or stressing the user before taking the measurement.
- If the thermometer moves from a warm/cold environment, allow it to acclimatise for 10–30 mins.
- The patient and the thermometer must be in the same room for at least 30 minutes.
- Do not hold the device in your hand for long periods of time to avoid heating up the sensors.
- Clean the forehead and probe before use; remove any hair, sweat or dirt.
- Wait 10 minutes after cleansing the skin or using cold/hot compresses.

## 8. FRONTAL MEASUREMENT

---

1. Insert the batteries (if using for the first time).
2. Press the "Measure/Mute" button to turn on.
3. Select the "Adult forehead" or "Child forehead" mode using the Mode/Memory button.
4. Place the probe in the centre of the forehead, 0–3 cm away.
5. Press and release "Measure".
6. After 1 second and an acoustic signal, read the value on the display.

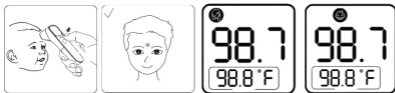**NOTE:** The frontal measurement may differ by  $\pm 0.5^{\circ}\text{C}$  from the internal temperature.

## 9. EAR MEASUREMENT

1. Switch on the device.
2. Select "Ear mode".
3. Remove the probe cover (if present).
4. Gently insert the probe into the ear canal.
5. Press and release the "Measure" button: the temperature is displayed in 1 second.

### TIPS FOR CORRECT EAR POSITIONING:

- For children under 1 year of age: pull the ear back slightly.
- For older children or adults: pull the ear upwards and backwards.
- Never force the probe.

### IMPORTANT:

- Always measure the same ear for consistency.
- A dirty ear or earwax can distort the reading.

## 10. AFTER MEASUREMENT

- Move the device away from your forehead/ear and read the value.
- Wait 10 seconds before taking a new measurement.
- You can consult the stored data using the memory mode.
- Clean the probe after each use.

**Do not self-diagnose or self-medicate based solely on the result. Consult your doctor if you have any doubts.**

## 11. READING AND INTERPRETING THE TEMPERATURE

The device uses colours and acoustic signals to quickly indicate whether the detected value is normal or feverish.

### FRONT/EAR METHOD:

Temperature (°C)	Indication	Signal
32.0–37.5	Green	1 long beep (normal)
37.6–38.0	Orange	6 short beeps (mild fever)
38.1–42.9	Red	6 short beeps (high fever)

## 12. "OBJECT" MODE (0–100 °C):

*Useful for measuring the temperature of surfaces, liquids or food without direct contact.*

1. Switch on the device.
2. Select "Subject" mode using Mod./Memory.
3. Point the thermometer towards the object at a distance of 0–3 cm.
4. Press "Measure" and wait for the beep.
5. Value shown on the display within 1sec.

**Do not measure liquids or shiny metals directly: point at the surface of the container.**

### 13. SILENT/ACTIVE MODE

---

Allows you to disable device sounds for quiet measurements (e.g., sleeping children).

1. Switch on the thermometer.
2. Press and hold the "Measure/Audio" button for approximately 5 seconds.
3. The speaker icon appears or disappears on the display.

### 14. INTERNAL MEMORY (40 MEASUREMENTS)

---

The device automatically stores the last 40 temperatures measured.

#### TO ACCESS THE MEMORIES:

1. Thermometer on or off.
2. Press and hold "Mode/Memory" for 5 seconds.
3. Scroll through the readings by pressing the same button several times.
4. "---" indicates no data recorded.

All memories are deleted when the battery is removed.

### 15. UNIT CHANGE °C / °F

---

Setting the correct unit of measurement prevents errors in interpreting the temperature.

1. Open the battery compartment.
2. Move the internal switch to °C or °F.
3. Close.

### 16. BATTERY AND POWER SUPPLY

---

The correct functioning of the thermometer depends on batteries with sufficient charge.

- Uses 2 x 1.5 V AAA batteries.
- Replace if the battery icon flashes or the device does not turn on.
- Remove the batteries if you do not use the device for more than 2 months.
- Automatic shutdown after ~13 seconds of inactivity.

### 17. TIPS FOR ACCURATE MEASUREMENTS

---

Small environmental or behavioural errors can alter the detected temperature.


- Know your normal personal reference body temperature.
- Always measure at the same spot (forehead, same ear).
- Wait 20 minutes if the person has been outdoors (very cold or hot).
- Wait 10 minutes after cleansing your forehead or applying compresses.
- Keep the device in the same room as the patient for at least 30 minutes.

### 18. CARE AND CLEANING

---

Proper cleaning ensures the accuracy and longevity of the device.

- Clean the probe and body of the thermometer with a soft cloth soaked in 70% alcohol.
- Allow to dry completely before reusing.
- Do not use abrasive detergents, solvents, benzene or immerse in water.
- Prevent liquids from entering the device or sensor.
- Do not scratch the LCD screen when cleaning.

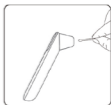## 19. TROUBLESHOOTING AND ERRORS

Temperature (°C)	Indication	Signal
The device does not switch on	Battery low or incorrectly inserted	Replace or reinsert correctly
Value too low	Dirty probe / excessive distance	Clean probe / move closer
Value too high	User comes from a warm environment	Wait 30 minutes at internal temperature
Red screen and long beep	Temperature out of range / system error	Measure at 5–40°C / contact customer service
"_ _ _"	No data in memory	-

## 20. TECHNICAL SPECIFICATIONS

Voice	Value
Product type	Digital forehead/ear thermometer
Regulations	ASTM E1965-98, ISO 80601-2-56
Power supply	2 × AAA 1.5 V
Body measurement field	32.0°C – 42.9°C (89.6°F – 109.2°F)
Object measurement field	0°C – 100°C (32°F – 199°F)
Accuracy	±0.2°C (body) / ±1°C (objects)
Memory	40 readings
Auto power off	13 seconds
Terms of use	5°C – 40°C; 15–95% RH
Storage	-20°C – 55°C; 0–95% RH
Weight and dimensions	60 g (without batteries); 143 × 35 × 41 mm

## 21. DEVICE SYMBOLS AND ELECTROMAGNETIC COMPATIBILITY

Symbol	Meaning
	Applied part type BF
	Compliant with European directives
	Consult the manual
	Keep away from sunlight
	Do not dispose of in household waste
	Handle with care

### APPENDIX: ELECTROMAGNETIC COMPATIBILITY INFORMATION GUIDE AND MANUFACTURER'S DECLARATIONS

**CAUTION:** • The infrared thermometer requires special precautions regarding electromagnetic compatibility and must be installed and commissioned in accordance with the electromagnetic compatibility information provided in the ACCOMPANYING DOCUMENTS. • Portable and mobile RF communications equipment may affect the infrared thermometer. • The infrared thermometer should not be used next to or stacked with other equipment. • Guide and manufacturer's declaration – electromagnetic emissions – for all EQUIPMENT AND SYSTEMS

## 22. WARRANTY - TERMS AND CONDITIONS

This product is covered by a 24-month warranty for manufacturing defects, starting from the date of purchase as proven by the receipt or invoice. If purchased with an invoice, the legal warranty is reduced to 12 months. The 24-month warranty **does not cover** damage resulting from normal wear and tear, cosmetic damage, accessories or consumables (e.g. batteries, heads or components subject to wear and tear). **Batteries are guaranteed for a period of 6 months.**

**WARRANTY EXPIRY:** The 12/24-month legal warranty shall expire in the following cases:

1. The product has been damaged due to improper use that does not comply with the instructions in the manual.
2. The product has been modified, tampered with or opened.
3. The malfunction is caused by insufficient maintenance of components, accessories or supplies (e.g. oxidation, immersion of electrical parts in liquids, residues blocking the sensor, corrosive liquid leaking from batteries).

**EXCLUSIONS FROM THE WARRANTY:** The following are excluded from the 24-month legal warranty:

1. The costs for replacing or repairing parts subject to wear and tear and those relating to routine maintenance of the product.
2. The costs and risks associated with transporting the product to and from the point of sale or the authorised centre for warranty management.
3. Damage resulting from incorrect installation or improper use not in accordance with the instructions in the manual.
4. Damage caused by natural disasters, falls, accidental impacts or environmental conditions incompatible with the product.
5. Defects that have a negligible effect on the performance of the product.

### LIMITATIONS OF LIABILITY:

The manufacturer, distributor and all parties involved in the sale of the product are not liable for economic losses or damages resulting from product malfunction. In accordance with current legislation, they are not liable for direct or indirect damage, loss of income, savings or other consequences arising from the use or inability to use the product and/or associated documentation (paper or electronic), including lack of service.

For further details on the assistance service, please refer to the Terms and Conditions of Sale available on the website [www.nuvitababy.com](http://www.nuvitababy.com).

**Guide and manufacturer's declaration - Electromagnetic emissions**

The infrared thermometer is intended for use in the electromagnetic environment specified below. The customer or user of the infrared thermometer must ensure that it is used in such an environment.

Emission test	Compliance
RF emissions CISPR 11	Group 1
RF emissions CISPR 11	Class B
Harmonic emissions IEC61000-3-2	Not applicable
Voltage fluctuations/flicker emissions IEC61000-3-3	Not applicable

**Manufacturer's guide and declaration - Electromagnetic immunity**

The infrared thermometer is intended for use in the electromagnetic environment specified below. The customer or user of the infrared thermometer must ensure that it is used in such an environment.

Immunity test	Test level IEC 60601-1-2	Level of compliance
Electrostatic discharge (ESD) IEC 61000-4-2	Contact $\pm 8$ kV $\pm 2$ kV, $\pm 4$ kV, $\pm 8$ kV, $\pm 15$ kV air	Contact $\pm 8$ kV $\pm 2$ kV, $\pm 4$ kV, $\pm 8$ kV, $\pm 15$ kV air
Fast electrical transients/bursts IEC 61000-4-4	Not applicable	Not applicable
Overvoltage IEC 61000-4-5	Not applicable	Not applicable
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	Not applicable	Not applicable
Industrial frequency magnetic field IEC 61000-4-8	30 A/m 50Hz / 60Hz	30 A/m 50Hz / 60Hz
Conducted RF IEC 61000-4-6	Not applicable	Not applicable
Radiated RF IEC 61000-4-3	10 V/m; 80 MHz - 2.7 GHz 80% AM at 1 kHz	10 V/m; 80 MHz - 2.7 GHz 80% AM at 1 kHz

NOTE UT is the AC mains voltage before the test level is applied.

# ELECTROMAGNETIC IMMUNITY

Manufacturer's guide and declaration - Electromagnetic immunity

The infrared thermometer is intended for use in the electromagnetic environment specified below. The customer or user of the infrared thermometer must ensure that it is used in such an environment.

Test frequency (MHz)	Band (MHz)	Service	Modulation	Maximum power (W)	Distance (m)	Test level IEC 60601-1-2 (V/m)	Compliance level (V/m)
385	380-390	TETRA	18 Hz pulse modulation	1.8	0.3	27	27
400	430-470	GMR5-460 FRS 460	FM ±5 MHz deviation 1 kHz sinusoidal	2	0.3	28	28
710							
745	704-787	LTE Band 13,17	217 Hz pulse modulation	0.2	0.3	9	9
780							
810							
870	800-960	GSM 800/900, TETRA 400, IDEN 820, CDMA 850, LTE Band 5	18 Hz pulse modulation	2	0.3	28	28
930							
810							
870	1700-1990	GSM 1800; CDMA 1900; GSM 1900; DECT; LTE Band 1, 3, 4, 25; UMTS	217 Hz pulse modulation	2	0.3	28	28
930							
2450	2400-2570	Bluetooth, WLAN 802.11 b/g/n, RFID 2450,	217 Hz pulse modulation	2	0.3	28	28
2450	2400-2570	Bluetooth WLAN, 802.11 b/g/n, RFID 2450,	217 Hz pulse modulation	2	0.3	28	28
810							
870	5100-5800	802.11 a/n WLAN	217 Hz pulse modulation	0.2	0.3	9	9
930							

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All rights reserved
#### Share Info GmbH
#### Shenzhen AOJ Medical Technology Co., Ltd.
#### Anteprima Brands International Ltd