+ nuvitaMED®

Bye Bite



- IT** Dopopuntura
- ES** Aparato contra picaduras de insectos
- EN** Insect Bite Healer
- FR** Appareil soulgeant les piqûres d'insectes
- DE** Insektenstichheiler
- PT** Curador de Picadas de Insetos
- GR** Θεραπευτής Τσιμπημάτων Εντόμων
- NO** Insektbittkurering
- DK** Insektbidsheler
- AL** Shëruesi i Picadhurave nga Insektet

2110

- IT** Manuale di istruzioni
- ES** Manual de instrucciones de uso
- EN** Instruction manual
- FR** Manuel d'instructions d'utilisation
- DE** Bedienungsanleitung

- PT** Manual de instruções de uso
- GR** Εγχειρίδιο οδηγιών χρήσης
- NO** Bruksanvisning
- DK** Instruktionsmanual
- AL** Manuali i udhëzimeve

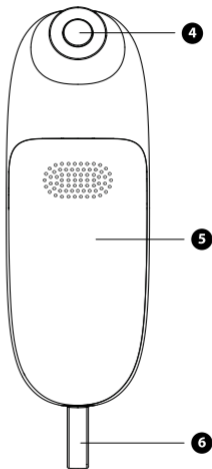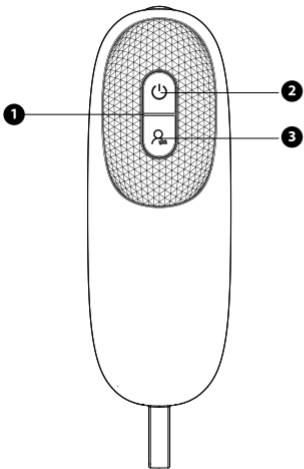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Thank you for purchasing Bye Bite Insect bite healer. Please read the Manual carefully before using the device, and put it in a safe and secure place for reference.

Intended Use: The Insect bite healer is intended for relieving itching and swelling caused by insect bites or stings by applying heat (local hyperthermia).

Target group: The Insect bite healer is suitable for people who want to relieve the itching and swelling caused by insect bites or stings. Over 3 years of age under supervision, can be used for self-treatment by anyone over 12 years of age.

Clinical benefits: Treatment and relief of itching and swelling caused by insect bites or stings.

Indication: Treatment with the insect bite healer is recommended for itching and swelling caused by insect bites or stings.

CONTRAINDICATION

- Do not use the device if you suffer from diabetes. Sensitivity to pain could, in certain circumstances, be reduced and diabetics may burn themselves.
- Do not use the device on people with disabilities, children under 3 years old or people with reduced sensitivity to heat. This includes people with skin alterations due to illness or scarred tissue in the application area, after taking pain relief medication, drinking alcohol or taking drugs.
- Do not use the device on acutely or chronically diseased (injured or irritated) skin (e.g. inflamed skin, whether painful or not, reddened skin, rashes, (e.g. allergies), burns, frostbite, bruises, swellings (without signs of an insect bite/sting), both open and healing wounds, and surgical scars where the healing process could be affected).
- Do not use the device if you are suffering from a fever.
- Do not use the device on sensitive areas of skin.

- Do not use the device if you have any sensory impairment that reduces the feeling of pain (e.g. metabolic disorders).
- Do not use the device if you are using lotions, creams and gels locally at the same time.
- Do not use the device if you suffer from persistently irritated skin due to long-term heat application on the same area of skin.

GENERAL WARNINGS

- Before using the device, consult your doctor if you have any skin conditions, serious illness (in particular if you have a propensity to thromboembolic conditions or recurrent malignant growths), unexplained chronic pain in any part of the body.
- Seek medical advice immediately if the symptoms of the sting persist after a few uses. Stop using the device.
- Seek medical advice immediately if the symptoms worsen whilst or after using the insect bite healer and stop using it.
- Seek medical advice immediately if it is a tick bite. The bite may result in transmission of pathogens such as tick-borne encephalitis (TBE) or borreliosis.
- Seek medical advice immediately if the first signs of increased itching, skin redness, swelling, dizziness, nausea, shortness of breath, formation of wheals or a drop or increase in blood pressure occur.
- Do not use near to the eyes, on the eyelids, mouth or on mucous membranes. Furthermore, the device must not be used on moist skin.
- Do not allow children to play with the device.
- Store the device out of the reach of children and pets.
- Keep packaging material away from children (risk of suffocation).
- Please note the general hygiene measures when using the device.
- The device is not a substitute for medical consultation and treatment. Always consult your doctor if you are experiencing any pain or are suffering from an illness.
- If the device does not work properly, or if you feel unwell or experience pain, stop using it immediately.

GENERAL PRECAUTIONS

- Do not wet the device.
- Do not use any detergents or solvents to clean and maintain the device. If liquid penetrates the device this could cause damage to the electronics or other device parts and lead to a malfunction.
- The device should only be used by one person; use by multiple people is not recommended.
- Do not expose the device to direct sunlight, store it in a dry place. Protect it from dust, chemicals and strong variations in heat.
- Protect the device from impacts, moisture, dust, chemicals, marked temperature fluctuations and nearby sources of heat (ovens, heaters).
- Before use, ensure that there is no visible damage to the device. In case of any doubt, do not use the device, and contact your retailer or the specified Customer Service address.

SAFETY NOTES

- If your skin or eyes come into contact with battery fluid, rinse the affected areas with water and seek medical assistance.
- Choking hazard! Small children may swallow and choke on batteries. Therefore, store batteries out of the reach of small children.
- Risk of explosion! Do not throw batteries into a fire.
- If a battery has leaked, put on protective gloves and clean the battery compartment with a dry cloth.
- Do not disassemble, open or crush the batteries.
- Observe the plus (+) and minus (-) polarity signs.
- Protect the batteries from excessive heat.
- Do not charge or short-circuit batteries.
- If the device is not to be used for a relatively long period, take the batteries out of the battery compartment.
- Use identical or equivalent battery types only.
- Always replace all batteries at the same time.
- Do not use rechargeable batteries!

NOTE ON ELECTROMAGNETIC COMPATIBILITY

- The device is suitable for use in all environments listed in these instructions for use, including domestic environments.
- The use of the device may be limited in the presence of electromagnetic disturbances. This could result in issues such as error messages or the failure of the device.
- Failure to comply with the above can impair the performance of the device.

DEVICE DESCRIPTION

The associated drawings are shown on page 2.

- | | |
|----------------------|----------------------------|
| 1. Device status LED | 4. Heating plate (MCH) |
| 2. Power button | 5. Battery compartment lid |
| 3. Activation button | 6. Silicone handle |

INSERTING THE BATTERIES

1. Push open the battery compartment lid **(5)**.
2. Insert the batteries included in delivery into the battery compartment as shown inside the battery compartment. When inserting the batteries, ensure that the correct polarity (+/-) is observed.
3. Slide the battery compartment lid **(5)** closed until you feel and hear it click into place.
4. Remove the batteries if you do not intend to use the device for a long time (over 3 months).

USAGE

WARNING

Before use, make sure that there is no insect sting remaining in the area that was stung. Remove the sting carefully before use. Use immediately after the insect bite or sting achieves the best results. If you wait too long to treat the area that has been stung or bitten and itching and swelling have already started, the insect bite healer can only have a limited effect. Nevertheless, treat the area with the insect bite healer. Usually, itching can be relieved and healing can be sped up.

1. Press the power button **(2)** to start the device. The device status LED **(1)** lights up green, an buzzer sounds and indicates to you that the device is ready to use.
2. For a 3-second application, place the heating plate (MCH) **(4)** of the device on the area that has been stung or bitten. Press the activation button once. The device status LED **(1)** lights up blue and signals that the application is starting. The treatment temperature of the device is $47\text{ }^{\circ}\text{C}\pm 2^{\circ}\text{C}$ ($116.6^{\circ}\text{F}\pm 3.6^{\circ}\text{F}$). This short application time is recommended for first use or use on sensitive skin.
3. For a 6-second application, place the heating plate (MCH) **(4)** of the device on the area that has been stung or bitten. Press the activation button twice in quick succession. The device status LED **(1)** lights up yellow and signals that the application is starting. The treatment temperature of the device is $50\text{ }^{\circ}\text{C}\pm 2^{\circ}\text{C}$ ($122^{\circ}\text{F}\pm 3.6^{\circ}\text{F}$). This application time is recommended for regular use on normal
4. After the end of the application, an buzzer sounds, heating of the heating plate (MCH) **(4)** stops automatically and the device status LED **(1)** lights up green again. Remove the heating plate (MCH) **(4)** from the area of skin once the signal sounds.

IMPORTANT

Depending on the sensitivity of the person using the device and of the area of skin, the treatment temperature may feel unpleasant and cause slight skin redness and, in rare cases, irritated skin. If the heat is too hot, stop treatment immediately.

5. If required, you can use the heat function of the insect bite healer on the sting/bite again after a break of 2 minutes or also use it immediately on another sting/bite in a different area. The maximum number of 5 applications per hour on the same treatment area must not be exceeded.

6. To turn off the device, press the on/off button **(2)**. The green device status LED **(1)** turns off. If there is no further operation within 5 minutes, the device will automatically shut down.
7. Before storing, make sure that the device is switched off and the heating plate **(4)** has cooled down.

Battery warning: Before the battery level becomes too low for other applications, the device status LED **(1)** starts to flash 3x and an acoustic signal sounds 3x. Replace both batteries soon.

CLEANING AND MAINTENANCE

Observe the following hygiene instructions to avoid health risks:

- The device is intended for multiple uses.
- It is recommended that the device is cleaned before first use and each additional use.
- The device must be switched off, allowed to cool down and the batteries removed each time before cleaning.
- Use only soft, dry cloths for cleaning the housing.
- Never use any substances for cleaning that could potentially be toxic if they came into contact with the skin or mucous membranes, or if they were swallowed or inhaled. Do not use any abrasive cleaning products and never submerge the device in water.
- Do not clean the device in the dishwasher!
- Avoid mechanical cleaning with brushes or similar, as this could result in irreparable damage.
- When using the device on multiple areas that have been stung and bitten, it is recommended that you clean the heating plate each time. This reduces the possibility of germs and bacteria being transferred from one area of skin to the next. Use a commercially available alcohol disinfection wipe to disinfect the heating plate.

DISPOSAL

- Do not repair or adjust the device yourself. Proper operation can no longer be guaranteed in this case.
- Do not open the device. Failure to comply with this instruction will void the warranty.
- Repairs must only be carried out by Customer Services or authorized retailers. Before making a claim, please check the batteries first and replace them if necessary.
- For environmental reasons, do not dispose of the device in the household waste at the end of its useful life. Dispose of the device at a suitable local collection or recycling point in your country. Dispose of the device in accordance with EC Directive - WEEE (waste Electrical and Electronic Equipment). If you have any questions, please contact the local authorities responsible for waste disposal.

DISPOSING OF THE BATTERIES

- Batteries must not be disposed of in the household waste. They may contain poisonous heavy metals and are subject to special refuse treatment.
- The codes below are printed on batteries containing harmful substances: **Pb** = Battery contains lead, **Cd** = Battery contains cadmium, **Hg** = Battery contains mercury.

WHAT IF THERE ARE PROBLEMS?

| Problem | Possible cause | Solution |
|--|---|--|
| The device does not switch on. | The battery installation is incorrect. | Install the battery over again. |
| | The batteries are flat. | Replace both batteries as described in these instructions for use. |
| The device no longer gets hot enough. | The battery level is too low. | Replace both batteries as described in these instructions for use. |
| Device outputs 3 acoustic signals / the device status LED (1) flashes – even after switching the device OFF and ON again. | Battery level is too low, meaning that the device may no longer be able to be operated. | Replace both batteries as described in these instructions for use. |
| Application cannot be started, even though the device status LED (1) lights up green. | The device needs to be restarted. | Press the power button again. |
| Device outputs 5 acoustic signals / the device status LED (1) flashes – even after switching the device OFF and ON again. | The device may be faulty. | Please contact Customer Services. |

TECHNICAL SPECIFICATIONS

| | |
|---|--|
| Type | Insect bite healer |
| Model | Bye Bite 2110 (IBR20) |
| Dimensions (L x W x H) | 109.5mm×34mm×24.8mm |
| Weight (without batteries) | 28g |
| Power supply | 2 x batteries, type AAA(LR03 1.5V) |
| Max. temperature | 50°C ±2°C (122°F ±3.6°F) |
| Applications | Sensitive skin: 3s,47°C±2°C(116.6°F±3.6°F) Regular skin: 6s,50°C±2°C(122°F±3.6°F) |
| Expected operating duration | Approx. 300 applications (with each charge) |
| Permissible storage and transport environment | -10°C to 50°C, relative humidity 30% to 85%, pressure 700hPa to 1060hPa |
| Permissible operating environment | 10°C to 40°C, relative humidity 30% to 85%, pressure 700hPa to1060hPa |

Essential Performance: Temperature control, Max temperature is 50°C ±2°C. In case of prolonged non-use, remove the batteries. **In accordance with our policy of continual product improvement, we reserve the right to make technical and aesthetic changes to the product.**

WARRANTY - TERMS AND CONDITIONS

This product benefits of a warranty of 24 months on material and manufacturing defects, starting from the date of purchase (see sales slip). The warranty of 24 months does not include damages

caused by the usual use of parties identified as “consumable” (e.g., batteries, brush heads or parts subject to usury). The legal guarantee of 24 months is void if: **1.** The product has undergone aesthetic damage due to improper use not in accordance with instructions in the manual. **2.** This product has been modified and/or tampered with. **3.** The cause of the failure was due to poor maintenance of the individual components and/or accessories and/or supplies (e.g. oxidation and/or scaling due to the retention of water or other liquids, sediment blocking the sensor, leak of corrosive liquid from batteries). The following is excluded from the legal guarantee of 24 months: **1.** Costs related to replacements and/or repair of parts subject to wear or costs for ordinary maintenance of the product. **2.** The costs and risks involved in transporting the product to and from the store where you purchased or otherwise authorized collection centre to receive the products under warranty. **3.** Damage caused by or resulting from improper installation or improper use not in accordance with the directions in the instruction manual. **4.** Damage due to natural disasters, accidental events or adverse conditions not compatible with the product. **5.** Defects that have a negligible effect on product performance. The manufacturer, distributor and all the parties involved in the sale do not assume any liability for losses and economic damage from any malfunction of the product. In accordance with current regulations the manufacturer, distributor and all the parties involved in the sale are not responding in any case for damages, including direct, indirect ones, loss of net income, loss of savings and additional damage and other details consequences going beyond the damage caused by the breach of warranty, contract, strict liability, wrongdoing or due to other causes, resulting from the use or inability to use the product and/or paper and electronic documents, including the lack of service. For further information on the help service visit the website www.nuvitababy.com

EXPLANATION OF SYMBOLS

The following symbols are used on the device, in these instructions for use, on the packaging and on the type plate for the device:

| | |
|--|------------------------------|
|  | Caution |
|  | Refer to instructions manual |

| | |
|---|---|
|  | Type BF Applied part |
|  | Disposal in accordance with the Waste Electrical and Electronic Equipment EC Directive – WEEE |
|  | Manufacturer |
|  | Date of Manufacture |
| LOT | Batch code |
| IP 42 | 4 Protected against solid foreign objects of 1,0 mm Ø and greater 2 Protected against vertically falling water drops when enclosure tilted up to 15° |
| EC REP | Authorized representative in the European Community |
| CE ₀₅₉₈ | CE mark |
| RoHS | RoHS mark |
| MD | Medical device |
|  | Indicates the entity importing the medical device into the local |
| UDI | Unique device identifier |

EMC INFORMATION

The ME EQUIPMENT or ME SYSTEM is suitable for home healthcare environments.

Warning: Don't near active HF surgical equipment and the RF shielded room of an ME system for magnetic resonance imaging, where the intensity of EM disturbances is high.

Warning: Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.

Warning: Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.

Warning: Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the equipment, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.

Even if that equipment complies with emission requirements, the Nuvitamed Bye Bite insect bite healer may be interfered with by other equipment.

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All rights reserved
Share Info GmbH
AOJ HEALTH Technology Co.,Ltd
Anteprima Brands International Ltd