nuvitamed

- IT** Aspiratore nasale elettrico
- ES** Aspirador nasal eléctrico
- EN** Electric nasal aspirator
- FR** Aspirateur nasal électrique
- DE** Elektrischer Nasenansauger
- PT** Aspirador nasal eléctrico
- GR** Ηλεκτρική ρινική αναρρόφηση
- NO** Elektrisk neseaspirator
- DA** Elektrisk næsesuger
- NL** Elektrische neusaspirator
- ALB** Aspirator elektrik nasal
- HU** Elektromos orrszívó
- CZ** Elektrická nosní odsávačka
- SI** Električni nosni aspirator
- BG** Электрически назален аспиратор
- SK** Elektrická nosová odsávačka
- RO** Aspirator nazal electric


2150

- IT** Manuale di istruzioni
- ES** Manual de instrucciones de uso
- EN** Instruction manual
- FR** Manuel d'instructions d'utilisation
- DE** Bedienungsanleitung

- PT** Manual de instruções de uso
- GR** Εγχειρίδιο οδηγιών χρήσης
- NO** Bruksanvisning
- DA** Instruktionsmanual
- AL** Manuali i udhëzimeve

INDEX

IT	Aspiratore nasale elettrico	004	SE	Elektrisk nässug	136
ES	Aspirador nasal eléctrico	010	FN	Sähköinen nenäimuri	142
EN	Electric nasal aspirator	016	EST	Elektriline ninaaspiraator	148
FR	Aspirateur nasal électrique	022	LV	Elektriskais deguna aspirators	154
DE	Elektrischer Nasenansauger	028			
PT	Aspirador nasal elétrico	034			
GR	Ηλεκτρική ρινική αναρρόφηση	040			
NO	Elektrisk neseaspirator	046			
DA	Elektrisk næsesuger	052			
NL	Elektrische neusaspirator	058			
ALB	Aspirator elektrik nasal	064			
HU	Elektromos orrszívó	070			
CZ	Elektrická nosní odsávačka	076			
SI	Električni nosni aspirator	082			
BG	Електрически назален аспиратор	088			
SK	Elektrická nosová odsávačka	094			
RO	Aspirator nazal electric	100			
HR	Električni nosni aspirator	106			
LT	Elektrinis nosies aspiratorius	112			
PL	Elektryczny aspirator do nosa	118			
SB	Električni nazalni aspirator	124			
GE	ელექტრო ცხვირის ასპირატორი	130			

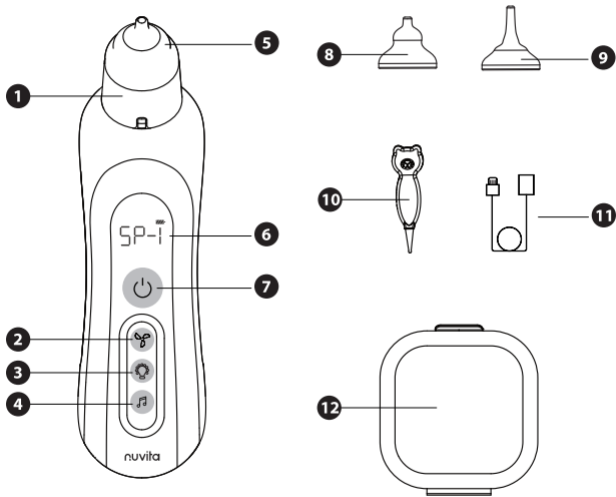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IMPORTANTE

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GENERAL WARNINGS

IMPORTANT

- Do not expose this product to high temperature, high humidity or direct sunlight for a long time.
- Please avoid severe pressure, collision or accidental fall of this product to the ground.
- This product is only suitable for absorbing nasal secretions. Do not use it for improper use such as absorbing water or skin.
- When the suction is significantly reduced or the motor sound is significantly reduced, please charge it in time.
- When using this product do not let the user hold his breath or swallow saliva and keep the baby's mouth open to avoid pressure imbalance in the ear.
- In case of any pain or discomfort during use, stop use immediately and seek medical attention.
- Do not put the suction head into the nasal cavity with excessive pressure.
- Please read the manual carefully before using the device and put it in a safe and secure place for reference.

DEVICE DESCRIPTION

The associated drawings are shown on page 2.

- | | | | |
|----|--------------------------|-----|--------------------|
| 1. | Collecting tank | 7. | Power button |
| 2. | Suction intensity button | 8. | Nozzle B |
| 3. | Coloured lights button | 9. | Nozzle C |
| 4. | Lullaby Button | 10. | Tweezers |
| 5. | Nozzle A | 11. | USB charging cable |
| 6. | LED Screen | 12. | Storage box |

FUNCTION DESCRIPTION

- **POWER BUTTON.** Press the power button **(7)** to turn on the nasal aspirator.
- **SUCTION INTENSITY BUTTON.** The suction has five intensity levels. Press the suction intensity button **(2)** once to enter level 1. Each subsequent press of the suction intensity button will cycle through the next level, up to level 5.
- **COLOURED LIGHTS BUTTON.** Press the coloured lights button **(3)** to turn on the coloured lights to entertain the baby during nasal suction, changing colour automatically.
- **LULLABY BUTTON.** Press the lullaby button **(4)** to play 3 soothing lullabies to aid relaxation and comfort for the baby during nasal suctioning.

SAFETY NOTES


IMPORTANT

- Fully charge the device before using it for the first time.
- There is no charging adapter attached. You can use an ordinary household charging adapter (Input 5V 1A; Output 3.7W).
- When charging, the power indicator on the LED screen **(6)** will flash.
- When the battery is full, the power indicator will stop flashing.
- Perform a cleaning cycle upon first use, following the instructions on page 6.


STEP 1

Press the silicone nozzle **(5)**, **(8)** or **(9)** into the collecting tank **(1)**.

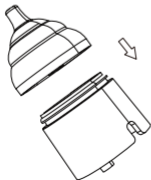STEP 2

Press the collecting tank with nozzle into the body of the device.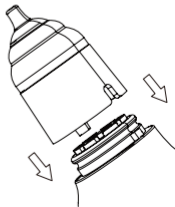

SAFETY NOTES

- Place the nasal aspirator close to the nasal cavity without leaving gaps, it will be easier to suck out nasal mucus.
- Before sucking hard nasal excrement or sticky nasal mucus, soften it, apply a hot towel to the nose or drip normal saline.
- Please pay attention to the nasal mucus collecting tank at any time. If there is too much nasal mucus some may overflow.
- Please shut down the machine in time when the mucus in chamber is full and clean it. Wipe the nasal mucus at the intake port and chamber with a cotton swab.
- Please empty the water and blow dry the nozzle and chamber after cleaning.
- Do not put the body into water or other liquids.
- Please use this product in the correct posture / method to avoid liquid being sucked into the host.

NOZZLE A



NOZZLE B

NOZZLE C

- **NOZZLE A.** The largest nozzle (5) can help infants and children suck pus type nasal mucus.
- **NOZZLE B.** Gourd type silicone suction nozzle (8), suitable for sucking shallow dirt.
- **NOZZLE C.** Funnel type suction nozzle (9), suitable for sucking deep dirt.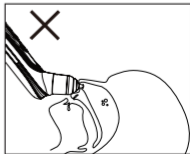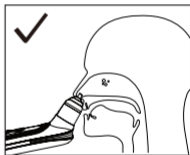

CLEANING AND MAINTENANCE

Follow these instructions to prevent health risks and product damage:

- **Clean the device before each use.** This is crucial for maintaining proper hygiene, especially in cases of bacterial and viral infections or when dealing with young children.
- **Body:** Use a soft dry cloth (or slightly damp) without adding detergents.
- **Internal Parts:** This device features the most advanced anti-reflux technology.

Procedure:

1. Fill a bowl with warm water
 2. Submerge the nozzle tip in water, with the tip pointing downward, and begin suctioning for 30 seconds. The water will pass from the nozzle to the reservoir and be expelled sideways, thus adequately cleaning all parts.
 3. Upon completing the cleaning, shake the device while keeping the nozzle tip pointing downward to ensure all residual water exits completely.
 4. Keeping the device with the nozzle tip pointing downward, disassemble the nozzle and reservoir and place the device vertically on a kitchen towel to facilitate the exit of water droplets that could penetrate the main body and cause irreparable damage.
- **Nozzles and Reservoir:** After washing as described in the previous point, the nasal nozzles and mucus collection reservoir can be rinsed with running water and sterilized by boiling (100°C) or using a steam sterilizer.
 - Never put the main body in a dishwasher or under a water jet
 - Never use potentially toxic cleaning substances that may come into contact with skin or mucous membranes, or that may be ingested or inhaled.
 - Do not use abrasive products and never immerse the device in water.
 - Avoid mechanical cleaning with brushes or similar, as it could cause irreparable damage.

DISPOSAL

- Do not repair or adjust the device yourself. Proper operation can no longer be guaranteed in this case.
- Do not open the device. Failure to comply with this instruction will void the warranty.
- Repairs must only be carried out by Customer Services or authorized retailers.
- For environmental reasons, do not dispose of the device in the household waste at the end of its useful life. Dispose of the device at a suitable local collection or recycling point in your country. Dispose of the device in accordance with EC Directive - WEEE (waste Electrical and Electronic Equipment). If you have any questions, please contact the local authorities responsible for waste disposal.

TECHNICAL SPECIFICATIONS

Type	Electric nasal aspirator
Model	2150
Suction Power	1st level: 25KPA, 2nd level: 35KPA, 3rd level: 45KPA, 4th level: 55KPA, 5th level: 65KPA
Battery type	Rechargeable lithium battery 3.7V; 450MAH; Voltage: 3.7V
Charging voltage	5V; Operating: 3.6~4.2V
Charging time	3 hours
Discharge duration	55 minutes
Noise level	1st level: 44DB, 2nd level: 50DB, 3rd level: 56DB, 4th level: 60DB, 5th level: 65DB
Material safety	BPA-Free, ABS PC, silicone, Non-Toxic

WARRANTY - TERMS AND CONDITIONS

This product is covered by a 24-month warranty for manufacturing defects, starting from the date of purchase proven by receipt or invoice. In case of purchase with invoice, the legal warranty is reduced to 12 months. The 24-month warranty **does not include** damage resulting from normal wear and tear of the product, aesthetic damage, accessories or consumables (for example batteries, heads or components subject to wear). **Batteries are guaranteed for a period of 6 months.**

INVALID WARRANTY: The 12/24-month legal warranty lapses in the following cases:

1. The product has been damaged due to improper use not in accordance with the instructions in the manual.
2. The product has been modified, tampered with or opened.
3. The malfunction is caused by insufficient maintenance of the components, accessories or supplies (e.g. oxidation, immersion of electrical parts in liquids, residues blocking the sensor, leakage of corrosive liquid from the batteries).

WARRANTY EXCLUSIONS: The following are excluded from the 24-month legal warranty:

1. The costs for the replacement or repair of parts subject to wear and those relating to ordinary maintenance of the product.
2. The costs and risks associated with transporting the product to and from the point of sale or the authorised centre for warranty management.
3. Damage resulting from incorrect installation or improper use not in accordance with the instructions in the manual.
4. Damage caused by natural disasters, falls, accidental impacts or environmental conditions incompatible with the product.
5. Defects that have a negligible impact on product performance.

LIMITATIONS OF LIABILITY:

The manufacturer, distributor and all parties involved in the sale of the product are not responsible for economic losses or damages resulting from the malfunction of the product. In accordance with the current legislation, they are not liable for direct, indirect, loss of income, savings or other consequences resulting from the use or inability to use the product and/or associated documentation (paper or electronic), including the lack of service.

For further details on the assistance service, please consult the Conditions of Sale available on the website **www.nuvitababy.com**.

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All rights reserved
Anteprima Brands International Ltd