nuvitamed


- IT** Aspiratore nasale elettrico
- ES** Aspirador nasal eléctrico
- EN** Electric nasal aspirator
- FR** Aspirateur nasal électrique
- DE** Elektrischer Nasenansauger
- PT** Aspirador nasal eléctrico
- GR** Ηλεκτρική ρινική αναρρόφηση
- NO** Elektrisk neseaspirator
- DK** Elektrisk næsesuger
- AL** Aspirator elektrik nasal

2150

- IT** Manuale di istruzioni
- ES** Manual de instrucciones de uso
- EN** Instruction manual
- FR** Manuel d'instructions d'utilisation
- DE** Bedienungsanleitung

- PT** Manual de instruções de uso
- GR** Εγχειρίδιο οδηγιών χρήσης
- NO** Bruksanvisning
- DK** Instruktionsmanual
- AL** Manuali i udhëzimeve

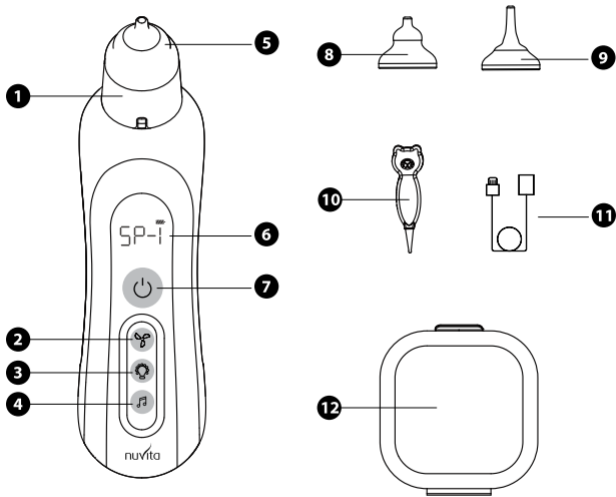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IMPORTANTE

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GENERAL WARNINGS

IMPORTANT

- Do not expose this product to high temperature, high humidity or direct sunlight for a long time.
- Please avoid severe pressure, collision or accidental fall of this product to the ground.
- This product is only suitable for absorbing nasal secretions. Do not use it for improper use such as absorbing water or skin.
- When the suction is significantly reduced or the motor sound is significantly reduced, please charge it in time.
- When using this product do not let the user hold his breath or swallow saliva and keep the baby's mouth open to avoid pressure imbalance in the ear.
- In case of any pain or discomfort during use, stop use immediately and seek medical attention.
- Do not put the suction head into the nasal cavity with excessive pressure.
- Please read the manual carefully before using the device and put it in a safe and secure place for reference.

DEVICE DESCRIPTION

The associated drawings are shown on page 2.

- | | | | |
|----|--------------------------|-----|--------------------|
| 1. | Collecting tank | 7. | Power button |
| 2. | Suction intensity button | 8. | Nozzle B |
| 3. | Coloured lights button | 9. | Nozzle C |
| 4. | Lullaby Button | 10. | Tweezers |
| 5. | Nozzle A | 11. | USB charging cable |
| 6. | LED Screen | 12. | Storage box |

FUNCTION DESCRIPTION

- **POWER BUTTON.** Press the power button **(7)** to turn on the nasal aspirator.
- **SUCTION INTENSITY BUTTON.** The suction has five intensity levels. Press the suction intensity button **(2)** once to enter level 1. Each subsequent press of the suction intensity button will cycle through the next level, up to level 5.
- **COLOURED LIGHTS BUTTON.** Press the coloured lights button **(3)** to turn on the coloured lights to entertain the baby during nasal suction, changing colour automatically.
- **LULLABY BUTTON.** Press the lullaby button **(4)** to play 3 soothing lullabies to aid relaxation and comfort for the baby during nasal suctioning.

SAFETY NOTES


IMPORTANT

- Fully charge the device before using it for the first time.
- There is no charging adapter attached. You can use an ordinary household charging adapter (Input 5V 1A; Output 3.7W).
- When charging, the power indicator on the LED screen **(6)** will flash.
- When the battery is full, the power indicator will stop flashing.


STEP 1

Press the silicone nozzle **(5)**, **(8)** or **(9)** into the collecting tank **(1)**.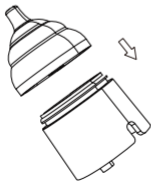


STEP 2

Press the collecting tank with nozzle into the body of the device.

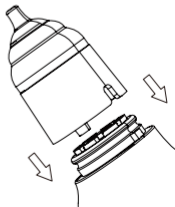
SAFETY NOTES

- Place the nasal aspirator close to the nasal cavity without leaving gaps, it will be easier to suck out nasal mucus.
- Before sucking hard nasal excrement or sticky nasal mucus, soften it, apply a hot towel to the nose or drip normal saline.
- Please pay attention to the nasal mucus collecting tank at any time. If there is too much nasal mucus some may overflow.
- Please shut down the machine in time when the mucus in chamber is full and clean it. Wipe the nasal mucus at the intake port and chamber with a cotton swab.
- Please empty the water and blow dry the nozzle and chamber after cleaning.
- Do not put the body into water or other liquids.
- Please use this product in the correct posture / method to avoid liquid being sucked into the host.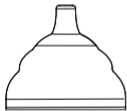

NOZZLE A


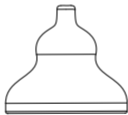
NOZZLE B

NOZZLE C

- **NOZZLE A.** The largest nozzle (5) can help infants and children suck pus type nasal mucus.
- **NOZZLE B.** Gourd type silicone suction nozzle (8), suitable for sucking shallow dirt.
- **NOZZLE C.** Funnel type suction nozzle (9), suitable for sucking deep dirt.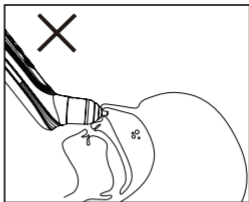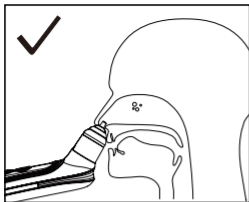

CLEANING AND MAINTENANCE


Observe the following hygiene instructions to avoid health risks:

- Clean the device before every use. This is important for the best hygiene especially in case of bacterial and viral infections or when the children are very young.

- **Body:** Use a soft dry (or slightly wet) cloth without adding any detergent.

- **Internal parts:** This device uses the latest anti back flow technology that allows the water and any liquid to go through the internal connections without damaging it. You can insert the tip of the nozzle in a container with some warm water and start to aspire it for 30 seconds. The water will go inside the internal parts and expelled aside.

- **Nozzles and Tank:** The nasal nozzles and the mucus collecting tank can be rinsed with running water and hot sterilised (100°C) in boiling water or with a steam sterilizer.

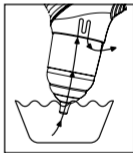- Never put the body in the dishwasher
- Never use any substance for cleaning that could potentially be toxic if it came into contact with the skin or mucous membranes, or if it's swallowed or inhaled.
- Do not use any abrasive cleaning products and never submerge the device in water.
- Avoid mechanical cleaning with brushes or similar, as this could result in irreparable damage.

DISPOSAL

- Do not repair or adjust the device yourself. Proper operation can no longer be guaranteed in this case.
- Do not open the device. Failure to comply with this instruction will void the warranty.
- Repairs must only be carried out by Customer Services or authorized retailers.
- For environmental reasons, do not dispose of the device in the household waste at the end of its useful life. Dispose of the device at a suitable local collection or recycling point in your country. Dispose of the device in accordance with EC Directive - WEEE (waste Electrical and Electronic Equipment). If you have any questions, please contact the local authorities responsible for waste disposal.

TECHNICAL SPECIFICATIONS

Type	Electric nasal aspirator
Model	2150
Suction Power	1st level: 25KPA, 2nd level: 35KPA, 3rd level: 45KPA, 4th level: 55KPA, 5th level: 65KPA
Battery type	Rechargeable lithium battery 3.7V; 450MAH; Voltage: 3.7V
Charging voltage	5V; Operating: 3.6~4.2V
Charging time	3 hours
Discharge duration	55 minutes
Noise level	1st level: 44DB, 2nd level: 50DB, 3rd level: 56DB, 4th level: 60DB, 5th level: 65DB
Material safety	BPA-Free, ABS PC, silicone, Non-Toxic

WARRANTY - TERMS AND CONDITIONS

This product benefits of a warranty of 24 months on material and manufacturing defects, starting from the date of purchase (see sales slip). The warranty of 24 months does not include damages caused by the usual use of parties identified as "consumable" (e.g., batteries, brush heads or parts subject to usury). The legal guarantee of 24 months is void if: **1.** The product has undergone aesthetic damage due to improper use not in accordance with instructions in the manual. **2.** This product has been modified and/or tampered with. **3.** The cause of the failure was due to poor maintenance of the individual components and/or accessories and/or supplies (e.g. oxidation and/or scaling due to the retention of water or other liquids, sediment blocking the sensor, leak of corrosive liquid from batteries). The following is excluded from the legal guarantee of 24 months: **1.** Costs related to replacements and/or repair of parts subject to wear or costs for ordinary maintenance of the product. **2.** The costs and risks involved in transporting the product to and from the store where you purchased or otherwise authorized collection centre to receive the products under warranty. **3.** Damage caused by or resulting from improper installation or improper use not in accordance with the directions in the instruction manual. **4.** Damage due to natural disasters, accidental events or adverse conditions not compatible with the product. **5.** Defects that have a negligible effect on product performance. The manufacturer, distributor and all the parties involved in the sale do not assume any liability for losses and economic damage from any malfunction of the product. In accordance with current regulations the manufacturer, distributor and all the parties involved in the sale are not responding in any case for damages, including direct, indirect ones, loss of net income, loss of savings and additional damage and other details consequences going beyond the damage caused by the breach of warranty, contract, strict liability, wrongdoing or due to other causes, resulting from the use or inability to use the product and/or paper and electronic documents, including the lack of service. For further information on the help service visit the website www.nuvitababy.com

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All rights reserved
Anteprima Brands International Ltd