# nuvitamed

<b>IT</b>	Aerosol a pistone	<b>PT</b>	Nebulizador de pistão
<b>ES</b>	Nebulizador de pistón	<b>GR</b>	Εμβολοφόρος νεφελοποιητής
<b>EN</b>	Piston nebuliser	<b>NO</b>	Stempelforstøver
<b>FR</b>	Nébuliseur à piston	<b>DA</b>	Stempelforstøver
<b>DE</b>	Kolbennebulisator		

5023

<b>IT</b>	Manuale istruzioni d'uso	<b>PT</b>	Manual de instruções de uso
<b>ES</b>	Manual de instrucciones de uso	<b>GR</b>	Εγχειρίδιο οδηγιών χρήσης
<b>EN</b>	Instruction manual	<b>NO</b>	Bruksanvisning
<b>FR</b>	Manuel d'instructions d'utilisation	<b>DA</b>	Bruksanvisning
<b>DE</b>	Bedienungsanleitung		

**INDEX**

---

<b>IT</b>	Aerosol a pistone	003
<b>ES</b>	Nebulizador de pistón	009
<b>EN</b>	Piston nebuliser	015
<b>FR</b>	Nébuliseur à piston	021
<b>DE</b>	Kolbennebulisator	027
<b>PT</b>	Nebulizador de pistão	033
<b>GR</b>	Εμβολοφόρος νεφελοποιητής	039
<b>NO</b>	Stempelforstøver	045
<b>DA</b>	Stempelforstøver	051
<b>NL</b>	Zuigervernevelaar	057
<b>ALB</b>	Aerosol me piston	063
<b>HU</b>	Dugattyús inhalátor	069
<b>CZ</b>	Pístový inhalátor	075
<b>SI</b>	Batni inhalator	081
<b>BG</b>	Бутален инхалатор	087
<b>SK</b>	Piestový inhalátor	093
<b>RO</b>	Aerosol cu piston	099
<b>HR</b>	Klipni inhalator	105
<b>LT</b>	Stūmoklinis inhaliatorius	111
<b>PL</b>	Inhalator tłokowy	117
<b>SB</b>	Klipni inhalator	123
<b>GE</b>	Kolbeninhalator	129
<b>SE</b>	Kolvinhalator	135
<b>FN</b>	Mäntäsumutin	141
<b>EST</b>	Kolb-inhalaator	147
<b>LV</b>	Virzuļa inhalators	153

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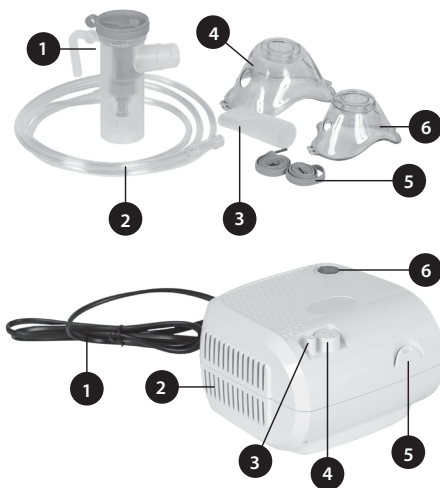
We are pleased you have purchased our product, and we thank you for your trust in us. Our goal is to achieve full satisfaction for our customers by offering them cutting-edge products for the treatment of respiratory diseases. Read these instructions carefully and retain them for future reference. Only use the accessory as described in this manual. This is a home medical device for nebulising and administering medication prescribed or recommended by your doctor after assessing the patient's overall condition.

## PRODUCT DESCRIPTION

1. Kit nebuliser
2. Air tube
3. Mouthpiece
4. Adult mask
5. Elasticated straps
6. Paediatric mask
7. Air filter x5 (not shown in image)

1. Power cord
2. Air vents
3. Air outlet / tube connector
4. Filter holder
5. Nebuliser holder
6. ON/OFF switch

## INSTRUCTIONS FOR USE

### NEBULISER

This device is an AC powered compressor air nebuliser system intended for medical purposes for use in home healthcare for use with the nebuliser kit to produce aerosolised particles of medications for respiratory therapy for both children and adults.

#### IMPORTANT

Read these instructions carefully before using this device.

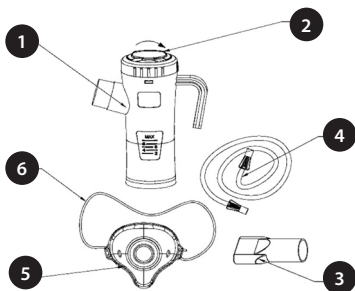
#### GENERAL INFORMATION

The compressor nebuliser system is for aerosol therapy and it is supplied with the kit nebuliser.

## INTEND USE

The nebuliser is intended to nebulize liquid medications in the form of aerosols.

	Parts	Material
1	Medicine container	PP
2	Lid with adjustable flow	PP
3	Mouthpiece	PP
4	Air tube	PVC
5	Mask (M/L)	PVC
6	Elastic	Spandex

## INSTRUCTIONS FOR USE

1. Connect one end of the hose to the air vent.
2. Open the top of the nebuliser by turning it counter-clockwise, then lift it up.
3. Pour the prescribed medication into the bottom of the nebuliser. The maximum filling volume is 8ml.
4. Close the nebuliser lid. The semicircle should face forward.
5. Connect the mouthpiece or mask to the nebuliser outlet.
6. Connect the free end of the air hose to the bottom of the container.
7. Turn on the device when you are ready to begin treatment.
8. To use with the mouthpiece, close your lips around the mouthpiece and inhale slowly. For the mask, place the mask over your nose and mouth and inhale slowly.

### CAUTION

Promiscuous use is not recommended.  
Keep away from heat sources.

## CAUTION

1. Use the device only for its intended purpose, as described in these instructions for use as a nebulization system with compressor for aerosol therapy. Follow your doctor's instructions. Any other form of use constitutes improper use and can be dangerous. The manufacturer cannot be held responsible for any damage caused by improper, incorrect or unreasonable use, or by connection of the unit to electrical systems that do not comply with safety standards.
2. Keep this manual for future reference.
3. Do not use the unit in the presence of anesthetics, flammable mixtures or oxygen.
4. The correct functioning of the device may be compromised by electromagnetic interference exceeding the limits indicated by the European standards in force. If this device interferes with another electrical appliance, remove it and plug it into a different electrical outlet.
5. In case of failure and/or malfunction, do not open it and do not attempt to repair it.
6. The correct functioning of the appliance may be compromised if original spare parts (e.g. filter) are not used.
7. When using any electrical appliance, some important safety measures must always be observed, including:
  - Use only original accessories and components;
  - Never immerse the unit in water as the device is not protected against water penetration;
  - Never touch the unit when your hands are wet or damp;
  - Do not leave the unit outdoors;
  - Place on a stable, horizontal surface during operation;
  - Make sure that the area of the ventilation openings is not obstructed;
  - Do not allow unsupervised children or people with physical disabilities to use the unit;
  - Do not turn off the unit by simply pulling the plug out of the electrical outlet.
8. Before connecting the device, make sure that the electrical rating indicated on the rating plate on the bottom of the unit corresponds to the main voltage and frequency.
9. Do not use simple or multiple adapters and/or extensions.
10. Unplug the device from the electrical outlet when not in use.
11. Follow the instructions for installing the device. The manufacturer cannot be held responsible for any damage caused by incorrect installation.
12. The power cord cannot be replaced by the user. In case the power cord is damaged, please contact our customer service for repair
13. The power cable must always be fully unwound to avoid dangerous overheating.
14. Before carrying out any cleaning or filter replacement operations, disconnect the device from the main power supply by unplugging it or turning off the main power switch.

15. Some parts of the unit are small enough for children to swallow. Do not let unattended children or people with physical disabilities have access to the unit.
16. At the end of its life the device must be disposed of according to local regulations in force.
17. Remember to:
  - Use this device only with medications prescribed by your doctor;
  - Use the accessories recommended by your doctor for therapy;
18. Check that the drugs prescribed by your doctor can be used with common compressor nebulizer systems for aerosol therapy.

## INSTRUCTIONS FOR USE

---

### IMPORTANT

Remove the compressor from carrying case before use.

Examine the unit before first use for shipping damage. During the treatment sit in an upright and relaxed position.

1. After unpacking the device, check the compressor and accessories for visible damage or defects such as cracks in the plastic housing that could expose the electrical components. Contact customer service to report any damaged products.
2. Before using the device, proceed with the cleaning operations as described in the "Cleaning and disinfection of accessories" section.

### ATTENTION

This device rated for intermittent use 30 minutes ON/ 30 minutes OFF.

## CLEANING AND MAINTENANCE

---

### CLEANING OF THE COMPRESSOR

Use a clean, dry cloth and mild detergents to clean the exterior of the unit.

### ATTENTION

When cleaning, make sure that the internal parts of the unit are not in contact with liquids and that the power cord is not connected.

### CLEANING AND DISINFECTION OF THE ACCESSORIES

After each treatment:

The medicine container can be cleaned by washing it with water and liquid soap and then disinfected by boiling it in water (max. 10 minutes). It is recommended to disinfect the mask and mouthpiece using cold disinfectant liquids. Dry the nebulizer using a clean cloth. After drying, connect it to the air outlet and turn on the compressor to check whether it works after cleaning.

**ATTENTION**

Do not boil the mouthpiece, air tube or masks. It is important to follow the cleaning instructions carefully to ensure optimal operation.

**REPLACEMENT OF THE NEBULISER**

The medicine container should be replaced after a long period of inactivity or if it is clogged with dried medication, dust, etc. With normal use, the container should last at least 6 months.

**REPLACEMENT OF THE FILTER**

The filter is to be replaced after approximately 30 hours of use, or when it appears dirty. Open the filter holder to replace the filter with a new one. Use original filters only.

**ATTENTION**

Do not use the device without the filter.

**TROUBLESHOOTING**

---

**The device does not turn on:**

1. Make sure the plug is firmly inserted into the electrical outlet.
2. Make sure the device has been operating within operating limits indicated in this instructional manual (30 minutes ON/ 30 minutes OFF).

**The device does not nebulise, or the nebulisation is weak:**

1. Make sure the ends of the air hose are securely attached to the main unit and nebuliser.
2. Check whether the medicine container is empty or too full (maximum filling is 8 ml)
3. Check if the nebuliser nozzle is clogged.
4. Check whether the deflector is inserted correctly.

**MAINTENANCE AND REPAIR**

---

**NEVER OPEN THE DEVICE**

There are no serviceable parts within the unit. The compressor requires no lubrication or maintenance.

## WARRANTY - TERMS AND CONDITIONS

---

This product is covered by a 24-month warranty for manufacturing defects, starting from the date of purchase proven by receipt or invoice. In case of purchase with invoice, the legal warranty is reduced to 12 months. The 24-month warranty **does not include** damage resulting from normal wear and tear of the product, aesthetic damage, accessories or consumables (for example batteries, heads or components subject to wear). **Batteries are guaranteed for a period of 6 months.**

### INVALID WARRANTY

The 12/24-month legal warranty lapses in the following cases:

1. The product has been damaged due to improper use not in accordance with the instructions in the manual.
2. The product has been modified, tampered with or opened.
3. The malfunction is caused by insufficient maintenance of the components, accessories or supplies (e.g. oxidation, immersion of electrical parts in liquids, residues blocking the sensor, leakage of corrosive liquid from the batteries).

### WARRANTY EXCLUSIONS

The following are excluded from the 24-month legal warranty:

1. The costs for the replacement or repair of parts subject to wear and those relating to ordinary maintenance of the product.
2. The costs and risks associated with transporting the product to and from the point of sale or the authorised centre for warranty management.
3. Damage resulting from incorrect installation or improper use not in accordance with the instructions in the manual.
4. Damage caused by natural disasters, falls, accidental impacts or environmental conditions incompatible with the product.
5. Defects that have a negligible impact on product performance.

### LIMITATIONS OF LIABILITY:

The manufacturer, distributor and all parties involved in the sale of the product are not responsible for economic losses or damages resulting from the malfunction of the product. In accordance with the current legislation, they are not liable for direct, indirect, loss of income, savings or other consequences resulting from the use or inability to use the product and/or associated documentation (paper or electronic), including the lack of service.

For further details on the assistance service, please consult the Conditions of Sale available on the website **[www.nuvitababy.com](http://www.nuvitababy.com)**.

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All rights reserved
### Foshan Hongfeng Co., Ltd
### Anteprima Brands International Ltd