nuvitamed

- IT** Apparecchio per aerosol portatile con compressore a pistone
- ES** Nebulizador para aerosol con compresor a pistón portátil
- EN** Portable compressor nebuliser for aerosol therapy
- FR** Nébuliseur à compresseur portable pour aérosolthérapie
- DE** Aerosolvernebler mit tragbarem Kolbenkompressor
- PT** Nebulizador compressor portátil para terapia com aerosol
- GR** Φορητός νεφελοποιητής συμπιεστή για θεραπεία με αεροζόλ
- NO** Bærbar kompressorforstøver for aerosolterapi
- DA** Bærbart aerosolapparat med stempelkompressor
- NL** Draagbaar aerosolapparaat met zuigercompressor
- ALB** Pajisje portative për aerosol me kompresor piston
- HU** Hordozható aeroszolkészülék dugattyús kompresszorral
- CZ** Přenosný aerosolový přístroj s pístovým kompresorem
- SI** Prenosna aerosolna naprava z batnim kompresorjem
- BG** Преносим аерозолен уред с бутален компресор
- SK** Prenosný aerosólový prístroj s piestovým kompresorom
- RO** Aparat portabil pentru aerosoli cu compresor cu piston
- HR** Prijenosni uređaj za aerosol s klipnim kompresorom
- LT** Nešiojamas aerozolio prietaisas su stūmokliniu kompresoriumi
- SB** Przenośne urządzenie do aerozolu z kompresorem tłokowym
- PL** Prenosni aparat za aerosol sa klipnim kompresorom
- GE** პორტატული აეროზოლის აპარატი დეჰუმანი კომპრესორით
- SE** Bärbar aerosolapparat med kolvkompressor
- FN** Kannettava aerosolite mäntäkompressorilla
- EST** Kaasaskantav aerosooliseade kolbkompressoriga
- LV** Pārnēsājama aerosola ierīce ar virzuļkompresoru

5030

- IT** Manuale istruzioni d'uso
- ES** Manual de instrucciones de uso
- EN** Instruction manual
- FR** Manuel d'instructions d'utilisation
- DE** Bedienungsanleitung

- PT** Manual de instruções de uso
- GR** Εγχειρίδιο οδηγιών χρήσης
- NO** Bruksanvisning
- DA** Brugsanvisning

INDEX

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Thank you for purchasing this Nuvita 5030 piston aerosol. Please read the operation manual carefully before using the product and keep it for future use. This product is a medical device. Be sure to follow a doctor's directions and use the device correctly. The nebulization characteristics of this product may be different depending on the properties of the drug. Especially with the use of a drug having a high viscosity the nebulization speed may be reduced. The nebulization speed may also be slower when the drug temperature is low.

SAFETY NOTES

Be sure to read the safety warnings before using the device.

WARNING

- For the type, dose and timing of taking the medication, be sure to follow a doctor's instructions.
- When using the device for the first time after purchase or after not using it for a long period of time, be sure to clean and disinfect the medicine container, the mouthpiece of the inhalation mask and the nosepiece.
- Clean and disinfect the medicine container, inhalation mask and mouthpiece after each use.
- Do not service the device while it is in use.
- Be sure to dry clean and disinfected parts and store them in a clean place.
- Do not place or attempt to dry the device, components, or any part of the nebulizer in the microwave.
- Keep the device out of reach of unsupervised children. The appliance may contain small parts that could be swallowed, and there may also be a risk of strangulation due to cables and tubes.
- Do not use non-original accessories such as medicine container, mask, nosepiece, mouthpiece.
- Do not wash the main unit and power plug with water.
- Do not immerse the main unit in water or other liquids.
- Do not cover the compressor with blankets, towels or any other type of covering during use. This could cause the compressor to overheat or malfunction.
- Do not use the device in locations where it may be exposed to flammable gases or vapours.
- Pentamidine is not an approved drug for use with this device.
- Always dispose of any remaining medication in the medicine container after each use.
- Do not leave the device or its parts exposed to extreme temperatures or changes in humidity. For example, do not leave your device in a vehicle during hot months and do not expose it to direct sunlight.
- Do not use or store your device in locations where it may be exposed to harmful fumes or volatile substances.
- Make sure the medicine container is clean before use.
- Not suitable for use in the presence of flammable anaesthetic mixtures with air, oxygen or nitrous oxide.

The device operates electrically and therefore may have a risk of electric shock:

- Do not use the compressor (main unit) or power plug if they are wet.
- Do not connect or disconnect the power plug from the electrical outlet with wet hands.
- Do not use or store the device in humid places, such as the bathroom. Use the device at temperature and humidity recommended.

Maintenance and conservation:

- Do not leave the liquid solution in the aerosol components. Rinse the nebulizer parts with clean, warm tap water after disinfection.
- Store the device and components in a clean and safe place.
- Do not open or repair the device yourself.

ATTENTION

- Do not pour water or other liquids on the compressor and the power plug. These parts are not waterproof. If liquid is spilled on these parts, immediately unplug the power plug and dry the liquid with gauze or other soft, absorbent material.
- Do not drop or subject the main unit or medicine container to strong shock. Do not disassemble, repair or modify the unit.
- Provide close supervision when this device is used by, on, or near infants, children, or dependent individuals. If the device is used continuously its life may be shortened.
- Limit use to 30 minutes at a time and wait 20 minutes before using the device again.
- Do not insert any objects into the compressor.
- Make sure the air filter is clean. If the air filter has changed colour or has been used for more than 60 days, replace it with a new original one.
- Make sure the medicine container is assembled correctly, the air filter is installed correctly, and the air hose is correctly connected to the compressor and medicine container. Air may leak from the air hose during use if the components are not installed correctly.
- Do not use the device if the air hose is bent.
- Do not add more than 8 ml of medicine to the medicine container.
- Do not use the device at temperatures above +40°C (+104°F).
- Do not tilt the medicine container so that the angle of the kit is greater than 45°. The drug may flow into the mouth.
- Do not shake the medicine container while using the device.
- Use only authorized parts and accessories. Parts and accessories not approved for use with the device do not meet the intended specifications or may damage the unit.
- To avoid injury to the nasal mucosa, do not compress the optional nose piece into the back of the nose. When using this device, you will hear noise and vibration caused by the pump in the compressor. You will also hear some noise caused by compressed air being released from the medicine container. This is normal and does not indicate a malfunction.
- When sterilizing parts by boiling, make sure that the water does not evaporate completely during boiling.
- Do not use the device while sleeping.
- Remove the power plug from the device after use.
- Disconnect the power plug from the electrical outlet before cleaning the device.
- Empty the medicine container and air tube and rinse them after each use. Then dry them with gauze or soft cloth.


SYMBOLS

Symbol	Explanation
	The caution symbol. This is a safety symbol used to highlight that there are specific warnings or precautions associated with the devices that are not otherwise found on the label. The symbol can also be used to indicate "caution, see instructions for use".
	Producer
	The symbol indicates that the appliance, at the end of its useful life, must be disposed of according to local regulations for separate waste collection.
	YOU MUST READ the OPERATION GUIDE symbol; indicates the need for the user to consult the instructions for use.
	This symbol indicates that the device complies with EC 60601-1 BF.
IP21	Protection against water ingress, means that the device can protect against water drops falling vertically at a maximum angle of 15°.
	Indicates the date the medical device was manufactured.
SN	This symbol must be accompanied by the manufacturer's serial number.
LOT	The lot number symbol: Indicate the manufacturer's batch code so that the batch can be identified.
EC REP	Indicates the authorized representative in the European Community.
CE	The CE mark symbol. This symbol certifies that a product has met the environmental, safety and consumer health requirements of the European Union.

PRODUCT CONTENTS


Please check the product. The following items are contained in the box. If any item is missing, contact the store where you purchased the product or your nearest retailer.

1. Compressor (unit principal)
2. Medicine container
3. Inhalation mask (adult)
4. Inhalation mask (child)
5. Mouthpiece
6. Air hose
7. Air filters (x5)
8. Connector
9. USB cable

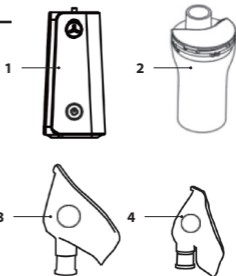NAME OF THE PARTS

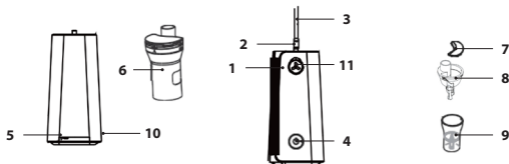1	Compressor (main unit)	6	Medicine container
2	Air outlet	7	Cover medicine container
3	Air pipe	8	Upper shell of medicine container
4	Power switch	9	Lower shell of medicine container
5	USB connector	10	USB interface
		11	Filter air vent and cover

COMPONENT NOMENCLATURE

 Make sure the mouthpiece, nose piece, mask and air filter are clean before use.

1. POWER SOURCE

It works directly with the battery. (Battery capacity 1200 mAh, charging power 5V 2.0 A max)

INDICATORS BRIGHT:

State of charge

- Red light: charging in progress
- Red light off: charging completed
- Red and blue lights on at the same time: The device is charging while in operation.

During use


- Blue light: The device is working
- Flashing blue light: low battery, the device is about to turn off automatically, charge it as soon as possible.

2. INTRODUCTION OF THE DRUG INTO THE CONTAINER

- Lift the mouthpiece, nose piece or mask from the medicine container and unscrew the top shell of the nebulizer.
- Caution: Be sure not to drop the deflector to the ground
- Place the correct amount of the prescribed medication into the nebulizer container.

3. CLOSING THE MEDICINE CONTAINER

- A. Screw the upper nebulizer shell to the lower nebulizer shell clockwise until it is securely closed.
- B. Install the desired inhalation accessories.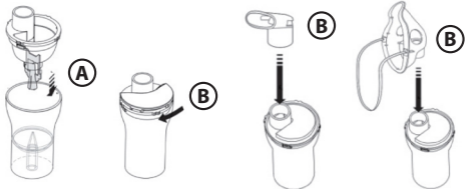

4. FIT THE TUBE OF THE AIR


Step 1: Rotate the air hose plug and push it firmly into the compressor air hose connector.

Keep the medicine container upright when assembling the air hose, otherwise the medicine may leak out of the medicine container.

CAUTION: Do not use the unit if the air hose is kinked.

Step 2: Rotate the air hose plug slightly and push it firmly into the air hose connector at the bottom of the medicine container.

Use the medicine container holder as a temporary support for the medicine container.

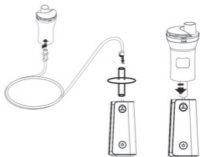HOW TO USE THE UNIT

1. Hold the medicine container as shown in the figure

Correct posture

CAUTION: Do not tilt the medicine container beyond a 45° angle, otherwise the medicine may flow into your mouth.

2. Turn on the unit

Press the power button for 1 second. When the compressor starts, nebulization begins.

3. Inhale the medicine following your doctor's instructions

Mouthpiece

Insert the mouthpiece into your mouth. Breathe in the medicine normally. Exhale normally through the mouthpiece

Mask

Place the mask over your nose and mouth. Pull the elastic strap over your head. Gently pull the strap to secure the mask over your nose and mouth. Inhale the medication. Exhale normally through the mask.

Nose piece

Inhale the medication through the nose piece as shown. Exhale through your mouth.

4. Turn off the unit

- Press the power button for 1 second. When the compressor stops the nebulization will also stop.

5. Disconnect the air hose from the medicine container

- Hold the air hose plug and gently pull it down
- Check the air hose. Make sure there is no condensation or moisture left inside.
If any condensation or moisture remains in the air hose, remove it by following the steps below:
 1. Make sure the air hose is still connected to the compressor air hose connector.
 2. Turn on the unit and the compressor will start and pump air through the air hose to expel moisture.
 3. Turn off the unit.

6. Disconnect the air hose from the compressor

- Hold the air hose plug and gently pull the air hose plug out of the air hose connector on the top side of the compressor.

HOW TO CLEAN THE UNIT AFTER INHALATION

 Be sure to clean and store the unit after inhalation.

1. Remove the inhalation attachment from the medicine container.
2. Disconnect the air hose from the medicine container.
3. Gently unscrew the top shell of the medicine container.
4. Discard the drug remaining.
5. Wash the parts in water.

6. Dry by hand or air in a clean environment using a soft, clean cloth.
7. Reassemble the medicine container and store it in a dry bag.

CAUTION: Clean the main unit case using a soft cloth moistened with water or a mild detergent. Do not use abrasive cleaners.

HOW TO DISINFECT

Disinfect the medicine cup and mouthpiece or nosepiece after the last treatment by following the instructions below:

Use a disinfectant compatible with aerosol mask disinfection.

- Immerse the parts in the disinfectant solution for the specified period
- Remove the parts and discard the solution
- Rinse the parts with clean, warm tap water, remove excess water and dry them in a clean environment.

ATTENTION

Never dry parts in the microwave. Never use an autoclave, EOG gas disinfection, or low-temperature plasma sterilizer to disinfect the device.

DEVICE STORAGE

- DO NOT store your device in locations exposed to high temperatures (over 60°C), low temperatures (below -20°C), high relative humidity (over 95%), or excessive amounts of dust. Never place heavy objects on or on the case
- Store the device and components in a clean and safe place. Hold the unit out from the scope some children.

MAINTENANCE

- DO NOT drop the device or subject it to other shocks or vibrations.
- Never dry parts in the microwave.
- Please note that parts other than masks and mouthpieces CANNOT BE DISINTEGRATED BY BOILING!
- Do not carry or leave the nebulizer with the medicine in the medicine container.

RESOLUTION OF THE PROBLEMS

Problem	Possible causes	How to correct
The device is unusually loud.	The air hose is connected incorrectly.	Secure the air filter cover correctly. Make sure the air filter cover is not blocked.
No power to device when pressing switch button	The battery is dead	Charge the battery

No atomization or low atomization speed when the device is turned on.	No medicine or too much medicine in the medicine container.	Add the correct amount of the prescribed medication
	The medicine container is not assembled correctly.	Make sure that the medicine container is assembled correctly, and that the inhalation accessory is connected correctly.
	The nozzle is blocked.	Clean and disinfect the medicine container to remove the blockage.
	The medicine container is tilted at an incorrect angle.	Hold the medicine container properly. Do not tilt it at an angle greater than 45 degrees.
	The air hose is connected incorrectly.	Make sure the air hose is connected correctly to the compressor and medicine container
	The air hose is kinked or damaged. The air hose is blocked.	Make sure the air hose is not kinked. Inspect the air hose for damage. Replace the air hose if damaged.
	The air filter is dirty.	Replace the air filter with a new, clean air filter.
The device is very hot.	The compressor is covered.	Do not cover the compressor with any type of cover during use.
	Continuous operation for over 30 minutes.	Limit use to 30 minutes at a time and wait 20 minutes before using the device again.

NOTE: If the unit does not nebulize normally after performing the above procedure, contact the nearest dealer where you purchased the device.

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Product name:	Piston nebulizer
Model:	5030
Standard accessories:	Nebulizer, air hose, adult mask, children's mask, mouthpiece, filters x5
Power source:	Built-in battery
Nebulization pressure:	40KPa-80KPa
Work environment:	Temperature: 10°C to 40°C Humidity: 30 to 80% relative humidity Pressure atmospheric: 70.0 KPa ~ 106.0 KPa

Conservation environment:	Temperature: -20 to 60°C Humidity: 10 to 95% relative humidity Pressure atmospheric: 70.0 KPa ~ 106.0 KPa
Device type:	Equipment
Battery charger:	5V, 2A
Noise level:	≥ 50dB (A)
Maximum capacity of the medicine container:	8 ml
Nebulization speed:	≥0,2 ml/min
Particle size:	MMAD: ≥3,5 µm +25%
Dimensions:	59 mm x 59 mm x 131 mm
Weight (excluding battery):	Approx. 250g (excluding medicine container and air tube)
Protection against harmful ingress of water or particles:	IP21

- The atomization speed may be reduced when used at low temperatures.
- Specifications and appearance may be changed without notice.

WARRANTY - TERMS AND CONDITIONS

This product is covered by a 24-month warranty for manufacturing defects, starting from the date of purchase proven by receipt or invoice. In case of purchase with invoice, the legal warranty is reduced to 12 months. The 24-month warranty **does not include** damage resulting from normal wear and tear of the product, aesthetic damage, accessories or consumables (for example batteries, heads or components subject to wear). **Batteries are guaranteed for a period of 6 months.**

INVALID WARRANTY: The 12/24-month legal warranty lapses in the following cases:

1. The product has been damaged due to improper use not in accordance with the instructions in the manual.
2. The product has been modified, tampered with or opened.
3. The malfunction is caused by insufficient maintenance of the components, accessories or supplies (e.g. oxidation, immersion of electrical parts in liquids, residues blocking the sensor, leakage of corrosive liquid from the batteries).

WARRANTY EXCLUSIONS: The following are excluded from the 24-month legal warranty:

1. The costs for the replacement or repair of parts subject to wear and those relating to ordinary maintenance of the product.
2. The costs and risks associated with transporting the product to and from the point of sale or the authorised centre for warranty management.
3. Damage resulting from incorrect installation or improper use not in accordance with the instructions in the manual.
4. Damage caused by natural disasters, falls, accidental impacts or environmental conditions incompatible with the product.
5. Defects that have a negligible impact on product performance.

LIMITATIONS OF LIABILITY:

The manufacturer, distributor and all parties involved in the sale of the product are not responsible for economic losses or damages resulting from the malfunction of the product. In accordance with the current legislation, they are not liable for direct, indirect, loss of income, savings or other consequences resulting from the use or inability to use the product and/or associated documentation (paper or electronic), including the lack of service. For further details on the assistance service, please consult the Conditions of Sale available on the website www.nuvitababy.com.

Guidance and manufacturer's declaration - Electromagnetic immunity

The mesh nebuliser is intended for use in the electromagnetic environment specified below . The customer or the user of mesh nebuliser should assure that it is used in such an environment.

Immunity Test	IEC 60601 Test Level	Compliance level	Electromagnetic environment- guidance
Electrostatic discharge GB/T17626.2	±6kV contact ±8kV air	± 6 kV contact ±2kV, ±4 kV, ±8kV, air	Floor should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%
Electrostatic Transient/ burst GB/T17626.4	Not applicable	Not applicable	Mains power quality should be that of a typical commercial environment
Surge GB/T17626.5	Not applicable	Not applicable	Mains power quality should be that of a typical commercial environment
Voltage dips ,short interruption and voltage variations on power supply input lines GB/T 17626.11	<5%UT, lasting 0.5 cycles (on UT, >95% temporary drop). 40%UT, lasting 5 cycles (on UT, 60% temporary drop). 70%UT, lasting 25 cycles (on UT, 30% temporary drop). ,5%UT, lasting 5S (on UT, >95% temporary Drop)	Not applicable	Mains power quality should be that of a typical commercial or environment. If the user of the mesh nebuliser requires continued operation during power mains interruptions, it is recommended that the mesh nebuliser be powered from an uninterruptible power supply or a battery.
Power frequency (40/60Hz) magnetic field GB/T 17626.8	3 A/m	3 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.

NOTE UT is the a.c. mains voltage prior to application of the test level

Guidance and manufacturer's declaration - Electromagnetic immunity

The mesh nebuliser is intended for use in the electromagnetic environment specified below . The customer or the user of mesh nebuliser should assure that it is used in such an environment.

Immunity Test	IEC 60601 Test Level	Compliance level	Electromagnetic environment- guidance
---------------	----------------------	------------------	---------------------------------------

Conducted RF GB/T 17626.6	3V 150kHz to 80MHz	3V	<p>Portable and mobile RF communications equipment should be used no closer to any part of the device, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommended Separation Distance</p> $d = 1.2 \sqrt{P}$ $d = 1.2 \sqrt{P} \text{ 80MHz} \sim \text{800MHz}$ $d = 2.3 \sqrt{P} \text{ 800MHz} \sim \text{2.5GHz}$ <p>Where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in meters (m). Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, should be less than the compliance level in each frequency range. Interference may occur in the vicinity of equipment marked with the following symbol:</p> 
Radiated RF GB/T 17626.3	3 V/m 80MHz ~ 2.5GHz	3 V/m	

Note 1: At 80MHz and 800MHz, the higher frequency range applies.
Note 2: These guidelines may not apply in all situations. Electromagnetic is affected by absorption and reflection from structures, objects and people.

- a. Stationary transmitters, such as base stations for wireless (peak/cordless) telephones and terrestrial mobile radios, amateur radio, AM and FM radio broadcasting, and television broadcasting, etc., whose field strength cannot be accurately predicted in theory. In order to assess the electromagnetic environment of fixed radio frequency transmitters, electromagnetic field surveys should be considered. If the measured field strength of the microgrid device is higher than the above radio frequency compliance level, the microgrid device should be observed to verify its normal operation. If abnormal performance is observed, supplementary measures may be necessary, such as re-adjusting the direction or position of the micro-mesh device.
- b. During the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m.

Recommended separation distance between Portable and mobile RF communications equipment and the Nebuliser

The Mesh Nebuliser is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the Mesh Nebuliser can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the Mesh Nebuliser as recommended below, according to the maximum output power of the communication equipment.

Rated maximum output of transmitter	Separation distance according to frequency of transmitter/m		
	150KH ~ 80MHz $d = 1.2 \sqrt{P}$	80MHz ~ 800MHz $d = 1.2 \sqrt{P}$	800MHz ~ 2.5GHz $d = 2.3 \sqrt{P}$
0.01	0.12	0.12	0.23
0.1	0.38	0.38	0.73
1	1.2	1.2	2.3
10	3.8	3.8	7.3
100	12	12	23

For transmitters rated at a maximum output power not listed above the recommended separation distance d in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (w) according to transmitter manufacturer.

Note 1: At 80MHz and 800MHz, the separation distance for the higher frequency range applies.
Note 2: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

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All rights reserved
WELLKANG LTD.,
SHENZHEN FITCONN TECHNOLOGY CO., LTD.
Anteprima Brands International Ltd