
nuvita

5075

5075 airy

MANUALE D'USO
USER MANUAL
MODE D'EMPLOI
MANUAL DE INSTRUCCIONES
BEDIENUNGSANLEITUNG

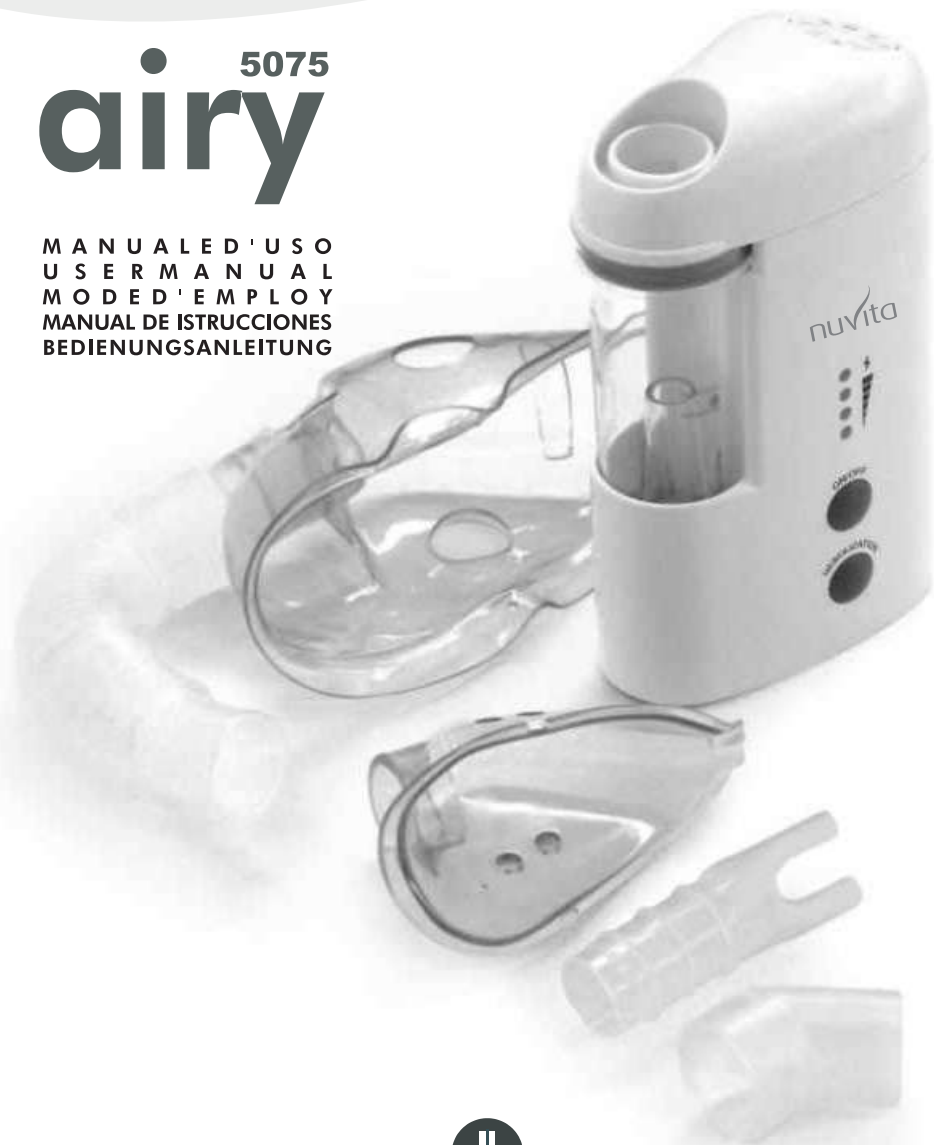SYSTEM FOR AEROSOL THERAPY - ULTRASONIC NEBULIZER - COMPACT SIZE 5075

SYSTEM FOR AEROSOL THERAPY - ULTRASONIC NEBULIZER - COMPACT SIZE
SISTEMA PER AREOSOL TERAPIA - AD ULTRASUONI - COMPATTO
SISTEMA PARA AEROSOL TERAPIA DE ULTRASONIDOS - COMPACTO
SYSTEME D'AÉROSOLTHÉRAPIE A ULTRASONS - COMPACT
SYSTEM FÜR AEROSOL THERAPIE MIT ULTRASCHALL - KOMPAKT
SYSTEEM VOOR SPIJTBUS THERAPIE ULTRASOUND - COMPACT

INDICE

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IMPORTED & DISTRIBUTED BY:
ANTEPRIMA BRANDS SRL - VIA RIVANI 99 - 40138 BOLOGNA - ITALY

MANUFACTURING:
VEGA TECHNOLOGIES INC. - 11F-13, 100
CHANG-CHUN ROAD, TAIPEI, TAIWAN

EU REPRESENTATIVE
SOLUTION GROUP S.P.A. - VIA SORMANI 1,
20095 CUSANO MILANINO (MI), ITALY

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INDEX

Introduction	9
Product Identification	10
Cautions	11
Cleaning Procedures	12
Operating Your Nebulizer	12
Troubleshooting	14
Product specification	14

INTRODUCTION

Your Airy 5075 should be used under the supervision of a licensed physician and/or a respiratory therapist.

Thank you for purchasing an Airy 5075. With the proper care and use, your Nebulizer will provide you with many years of reliable treatments.

Your Hi-Med Airy 5075 has been designed to offer compact, simple and lightweight (under 350g) convenience while receiving inhalation treatments. This unit requires ordinary tap or distilled water and your physician-prescribed medication to create an effective high frequency mist. Treatments can be administered discreetly thanks to advanced ultrasonic technology making your Nebulizer virtually silent during operation. An easy airflow volume control with triple settings allows full control of treatment levels as recommended by your physician and/or respiratory therapist.

This unit operates on standard AC power. Treatments are delivered quickly, safely and conveniently making this unit ideal for all ages.


We encourage you to thoroughly read this guidebook to learn about the feature of your Airy 5075. Together with your physician and/or respiratory therapist, you can feel comfortable and confident knowing that you are obtaining the most effective inhalation treatments for your respiratory condition.

PLEASE READ THIS GUIDEBOOK COMPLETELY BEFORE OPERATING THIS UNIT
The Airy 5075 is an ultrasonic nebulizer that is intended for use in the threatment of asthma, COPD and other respiratory ailments in which an aerosolized medication is required during therapy. The Airy 5075 is not intended for use with Pentamidine.


PRODUCT IDENTIFICATION


Power socket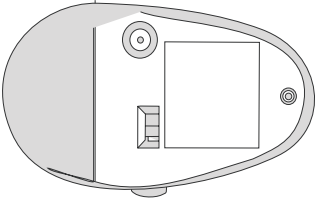


Mouth nozzle

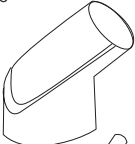
Nose nozzle

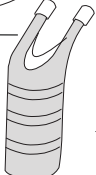
Child/adult mask

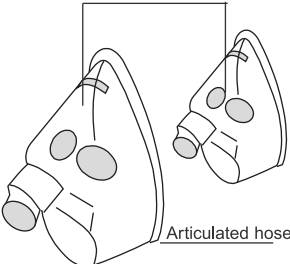Articulated hose

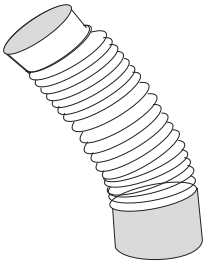Top cover

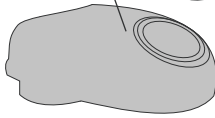Filter cover

Air filter


Medicine compartment

Fan

Green led indicating aerosol operation
Red led indicating run down battery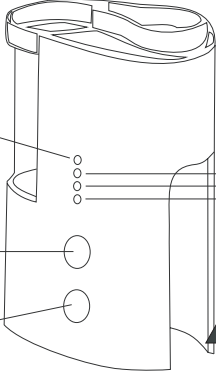

Vaporizer level indicator

ON/OFF button

Vaporization control button

Main body

Optional

CAUTIONS

Product Cautions

Please use general safety precautions when operating your Airy 5075.

This unit should be used only for (1) its intended purpose as described in this guidebook, and (2) with medications only under the supervision and instruction of your physician.

READ THE FOLLOWING BEFORE USING

- To avoid electrical shock: Keep unit away from water
- Do not handle the unit or power cord with wet hands
- Do not immerse the AC adapter or unit in liquid
- Do not use while bathing
- Do not reach for a unit that has fallen into water – immediately unplug the unit.
- Do not use the unit if it has any damaged parts (including power cord or plug), if it has been submersed in water or dropped. Promptly send the unit for examination and repair.
- The unit should not be used where flammable gas, oxygen or aerosol spray products are being used.
- Keep the air vents open. Do not place the unit on a soft surface where the openings can be blocked.
- If the water reservoir or medicine cup is empty, do not attempt to operate the unit .
- If any abnormality occurs, discontinue use until the unit has been examined and repaired.
- The unit should not be left unattended while plugged in.
- Do not use solutions other than water in the tank. It may damage the transducer.
- Do not tilt or shake the unit when in operation.
- Opening the top cover during operation may cause a severe injury.
- Disconnect the unit from the electrical outlet before cleaning, filling and after each use.
- Do not use attachments unless recommended by the manufacturer.
- The unit should not be used for more than 30 minutes at a time.
- Wait 30 minutes before reusing the unit.

Operating Cautions

- Keep your eyes away from the output of medication mist.
- Change the water after each use.
- The maximum capacity of the medicine cup is 8 ml. And should not be overfilled.
- Dilute dense medications or medication volume 2ml or less with distilled water or saline solution to obtain a volume of at least 4 ml.
- Do not use this unit while operating a vehicle.
- If any discomfort or abnormality occurs, stop using the unit immediately.

Storage Cautions

- Do not store the unit in direct sunlight, high temperature or humidity.
- Keep the unit out of the reach of small children.
- Always keep the unit unplugged while not in use

Cleaning Cautions

- Check the medication cup, mouth nozzle and air filter before each use. Dirty or worn parts should be replaced.
- Do not immerse the unit in water. It may damage the unit.
- Disconnect the unit from the electrical outlet before cleaning.
- Clean all necessary parts after each use as instructed in this guide.

CLEANING PROCEDURES

It is important to clean your Airy 5075 before the first time it is used and after each use.

Remember: Disconnect the unit from the electrical outlet and never immerse the unit in water

Cleaning the parts:

The mouth nozzle, nose piece, extension tube, mask, top cover, medicine cup, airflow cover, fan and air filter should be cleaned with a mild detergent or commercially available disinfectant. Thoroughly rinse each part and allow to completely air dry before using.

Cleaning the main unit:

The main unit should be cleaned with a soft cloth moistened with a non-abrasive cleanser and water.

OPERATING YOUR NEBULIZER

Follow the previous cleaning instructions prior to using your Airy 5075 for the first time or after it has been stored for an extended period.

Remember: Always unplug this product and turn off and switch at bottom before cleaning and filling and after each use.

To fill the water tank

1. Remove the top cover by pulling straight up. Fig. 1. Remove the medicine cup if applicable. Fig. 1
2. Select either tap or distilled water* that has a temperature range of 68°F – 86°F (20°C – 30°C). - Distilled water is preferred.
3. Fill the water reservoir with tap or distilled water to the indicated level (between the two lines located inside the water reservoir), Fig. 2-3.

Remember: Do not use solutions other than water in the reservoir. It may damage the transducer – Fill to the correct level.

Note: The use of cool water (below 68° F or 20°C) will result in a lower rate of nebulization.

The use of warm water (above 86°F or 30°C) will result in a higher rate of nebulization.

To fill the medicine cup

1. Place the medicine cup inside the water reservoir .
2. Pour the medicine directly into the medicine cup.
3. Replace the top cover

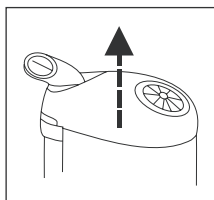
Fig. 1

Fig. 2-3

Remember: The maximum capacity of the medicine cup is 8 ml. and should not be overfilled. Dilute dense medication or medication volumes 2 ml. or less with distilled water or saline solution to obtain a volume of at least 4 ml. Only use medications under the instructions of your physician.

To connect the AC adapter

1. Insert the adapter plug into the Unit's AC port, Fig.4.
2. Plug the adapter into a standard AC outlet.

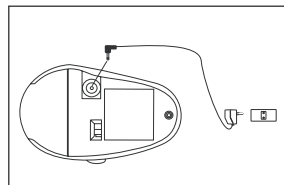
Fig. 4

To start the unit

1. Place the unit on a solid, flat, dry surface making sure that the water reservoir and medicine cup have been properly filled and mouth nozzle is properly attached to the top cover.
2. Turn the switch at bottom of unit to "ON" position. The LED indicator should be lit.
3. Press the ON/OFF button.

Note: Your unit has been designed to automatically output the highest level of airflow. The Airflow Volume Indicator Light will illuminate at the highest setting.

To adjust the airflow and nebulizer rate

1. Your Ultrasonic Nebulizer has 3 airflow settings: Maximal, Medium and Minimum.
2. Based on instructions from your physician and/or respiratory therapist, press the Nebulization rate control button to adjust the airflow volume. The Airflow Volume Indicator Light will show the level (Maximal, medium or minimum) selected. Fig. 5.

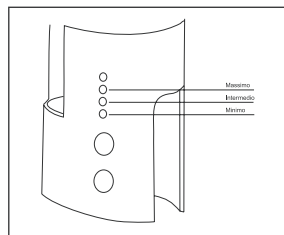
Fig. 5

To begin a treatment

1. Follow the instructions provided by your physician and/or respiratory therapist.
2. Your Nebulizer is designed to start at a lower atomization rate in the beginning and will become stronger after a few minutes.

Remember: Keep your eyes away from the output of medication mist. If any discomfort or abnormality occurs, stop using the unit immediately. Do not tilt or shake the unit when in operation. Opening the top cover during operation may cause a severe injury.

To end a treatment

1. When the treatment has ended, press the ON/OFF button stop operation. Turn the switch at bottom to "OFF/CHARGE" position. Otherwise, your unit will automatically shut off approximately 10 minutes of continuous use.
2. Unplug the adapter from the outlet and detach the adapter plug from the unit's AC port.
3. Allow the unit to cool – approximately 20-30 minutes.

Remember: The unit should not be used for more than 30 minutes at a time. Wait 30 minutes before reusing the unit.

After every use

1. Allow the unit to completely cool.
2. Remove the top cover.
3. Carefully remove the medicine cup and pour out remaining medication.
4. Empty the water reservoir.
5. Wipe out the water reservoir and transducer with a soft, lint-free cloth.
6. Follow the cleaning procedures provided in this guidebook.

TROUBLESHOOTING

If any abnormality occurs during use, please check and correct the following:

Condition	Correction
Unit does not operate when ON/OFF button is pressed	<p>Check the AC adapter connection to the outlet and/or check the AC plug to the unit's AC port.</p> <p>Check the switch at bottom is turned on.</p> <p>Check that the water reservoir is filled to the correct level.</p> <p>If using with optional battery pack, check that the battery is completely charged. Check that the battery is mounted correctly to the unit</p>
No misting or low rate of misting	<p>Check that there is medication in the medicine cup.</p> <p>Check the medication cup for damage.</p> <p>Check that the transducer is clean.</p>
Fan is not working properly	<p>Check that the fan is firmly and properly mounted on the motor shaft.</p>

PRODUCT SPECIFICATIONS

Unit specifications

Name and model number	Airy 5075 NB-08L
Power	100-120VAC or 220-240VAC 50/60Hz
Power Consumption	Approximately 10 W
Ultrasonic Frequency	2,5 MHz
Nebulization	3-speed adjustable 0,2 to 0,7 ml/minute depending on medicine density
Partical Size	5 µm MMAD
Medicine Cup Capacity	Approximately 8 ml maximum
Timer	Approximately 10 minutes
Dimensions	8,8 x 5,3 x 14,3 cm
Wheight	Approximately 230 g (350g with battery pack)
Accessories	AC adapter, Adults & Child masks, Extension tube, Carrying Case, 5 Reusable Medicine Cups

WARNING: The  symbol on this product means that it's an electronic product and following the European directive 2000/96/CE the electronic products have to be dispose on your local recycling centre for safe treatment.

IMPORTED & DISTRIBUTED BY:
ANTEPRIMA BRANDS SRL
VIA RIVANI 99 - BOLOGNA - ITALY

MANUFACTURING:
VEGA TECHNOLOGIES INC. - 11F-13, 100
CHANG-CHUN ROAD, TAIPEI, TAIWAN

EU REPRESENTATIVE:
SOLUTION GROUP S.P.A. - VIA SORMANI 1,
20095 CUSANO MILANINO (MI), ITALY

Declaration of Conformity

The product is in conformity with the essential requirements of the directive 1993/42/EC and carries the CE mark accordingly.

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