

1205_25_Manual_v1.0.0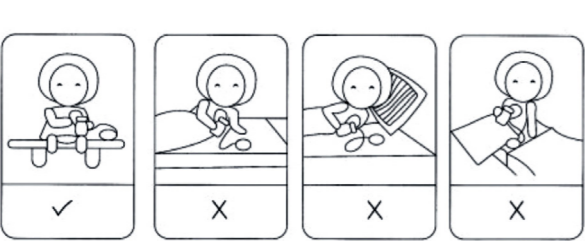

IMPORTED / DISTRIBUTED BY:
Anteprima Brands International Ltd
1, Ferris Bldg, St.Luke Street PTA1020 Gwardamangia, Malta - Europe. info@nuvitababy.com | www.nuvitababy.com
All rights reserved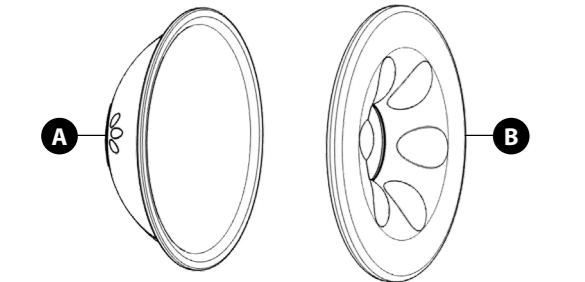

EN

1. INTRODUCTION

This manual breast milk collector offers a discreet milk drops collection sitting comfortably inside the bra with a soft silicone cover for a comfortable fit. Compact and portable, it's easy to carry for travel or work. All parts detach quickly for thorough cleaning and reliable hygiene.

2. PARTS

- A. Silicone cup x2
- B. Cup body x2

3. PREPARATION & USE

- NOTE:** Wash hands thoroughly and sterilize all components before use. Handle with care as parts may still be hot after sterilization.
- 1. Attach the silicone cup **(A)** to the cup body **(B)**, pressing firmly around the rim to ensure a full seal.
- 2. Confirm all parts are installed correctly before use.

4. TIPS FOR EFFECTIVE MILK COLLECTION

Choose a time when milk flow is naturally stronger, usually in the morning, before or after the first feeding. If you feel pain, stop and consult a lactation specialist.

5. CLEANING & STERILIZATION

- All parts can be sterilized. A professional sterilizer is recommended.
- Clean silicone parts with a soft sponge to avoid scratches.
- If boiling for sterilization, place parts in boiling water for 3 minutes. Prevent direct contact with the pot bottom to avoid damage.

6. STORAGE GUIDELINES

- Pour the collected milk in a clean baby bottle for storage.
- Gently swirl to mix separated fat.
- Do not heat breast milk with microwave or boiling water.

7. WARNING

- Intended for personal use only. Do not share with other users.
- Inspect all components before each use. Do not use if parts are damaged, deformed, cracked, or show signs of wear.
- Do not use while sleeping or during activities that may cause excessive pressure on the product.
- Discontinue use immediately if pain, irritation, redness, or discomfort occurs and consult a healthcare professional or lactation specialist.
- Do not modify the product or use non-original replacement parts.
- Keep away from open flames, heat sources, and direct sunlight.
- Sterilization and cleaning must follow the instructions provided. Incorrect sterilization may compromise product safety and hygiene.

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