nuvitalum

MATERNO *SMART* X


IT	Tiralatte elettrico
ES	Sacaleches eléctrico
EN	Electric breast pump
FR	Tire-lait électrique
DE	Elektrische Milchpumpe
PT	bomba tira-leite elétrica
GR	Ηλεκτρική αντλία στήθους
NO	Elektrisk brystpumpe
DA	Elektrisk brystpumpe


1287CX / 1288CX

IT	Manuale istruzioni d'uso	PT	Manual de instruções de uso
ES	Manual de instrucciones de uso	GR	Εγχειρίδιο οδηγιών χρήσης
EN	Instruction manual	NO	Brugsanvisning
FR	Manuel d'instructions d'utilisation	DA	Brugsanvisning
DE	Bedienungsanleitung		

1287CX MATERNO SMART X

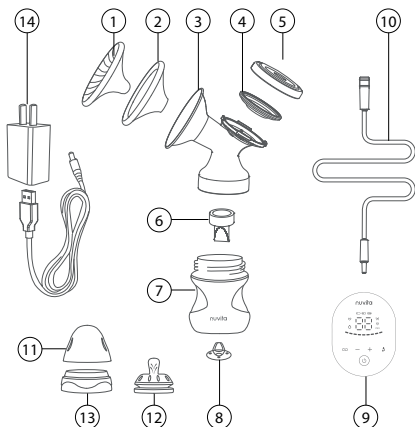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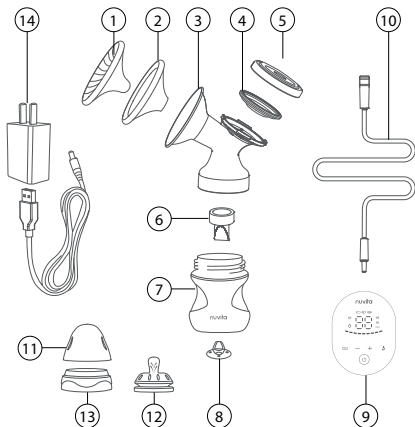CENTRALINA DEL TIRALATTE

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BREAST PUMP CONTROL UNIT

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Dear Customer, you bought one of the most reliable breast pumps available in the market. We studied this device to be as much user friendly as possible but, we are afraid, you still must spend a small amount of time in reading these instructions in order to get 100% of the benefits this product can bring you. Keep this manual for future reference.

PRECAUTIONS

- Please read the instruction manual carefully before using the product.
- This product must be used by adult only with complete attention during the entire process.
- Do not use the power adapter other than the provided.
- Do not warp or knot the connecting tube (Package Content No. 10) during the using process.
- Please put the product or dismantled parts out of baby's reach.
- Do not dismount Breast Pump Control Unit (Package Content No. 9) in any condition; it shall be repaired or replaced by the manufacturer or professional.

PACKAGE CONTENT

Before assembly and use the electric breast pump to make sure that inside the packaging there are the following accessories, refer to your shop in case of any missing parts:

1287CX MATERNO SMART X

1. Dust-proof cover
2. Breast cup
3. Breast Pump Body
4. Anti-spill silicone
5. Anti-spill cover
6. Suction Valve
7. Nuvita Baby Bottle (150ml)
8. Nuvita Bottle Silicone Valve
9. Breast Pump Control Unit
10. Connection Tube
11. Dust proof feeding bottle cover
12. Bottle Teat
13. Feeding bottle ring
14. Power Adapter
15. Velvet Bag (not shown in the diagram)

1288CX MATERNO SMART X DOPPIO

1. Dust-proof cover (2x)
2. Breast cup (2x)
3. Breast Pump Body (2x)
4. Anti-spill silicone (2x)
5. Anti-spill cover (2x)
6. Suction Valve (2x)
7. Nuvita Baby Bottle (150ml) (2x)
8. Nuvita Bottle Silicone Valve (2x)
9. Breast Pump Control Unit
10. Connection Tube (2x)
11. Dust proof feeding bottle cover (2x)
12. Bottle Teat (2x)
13. Feeding bottle ring (2x)
14. Power Adapter
15. Velvet Bag (not shown in the diagram)

INSTRUCTION FOR CLEANING

- Before cleaning and sterilization, dismount the parts in contact with the breast or breast milk (apart from Breast Pump Control Unit (Package Content No. 9)).
- Do not clean the Breast Pump Control Unit (Package Content No. 9) in the water.
- Do not sterilize the Breast Pump Control Unit (Package Content No. 9) in the steam sterilizer.
- Do not hot sterilize the connection pipe. Cold sterilization only. Otherwise cleaning with warm soapy water and then rinse with sterile water. (Package Content No. 10).
- Clean the parts immediately after each use to avoid the residual milk and bacteria growth.
- Put the cleaned parts on the dry clean towel at the shady and cool place.

PRODUCT FEATURES

A rechargeable electric breast pump model for maximizing milk extraction process. It's designed to simulate the suction rhythm of baby.

- This device is provided with an anti-reflux protection system which prevents the breast milk to return to the control unit and damaging it during the milk extraction process.
- Your Materno Smart is portable (AC/DC and Rechargeable Battery operated) so suitable for travelling and perfect for breast pumping activity at work.

INFORMATION

This electric breast pump model is designed to support breastfeeding mothers by simulating a baby's natural sucking rhythm. This electric breast pump is specifically designed to help mothers express breast milk efficiently and comfortably, even on the go, offering an effective and comfortable milk collection experience. It features 9 adjustable intensity levels, 3-stage Careflo® technology, and the NipFit® feature.

CAREFLO® TECHNOLOGY:

Unlike traditional 2-phase breast pumps, Careflo® introduces a third anti-stasis drainage phase alongside the standard stimulation and expression stages. This helps to:

- Promote a more consistent milk supply.
- Prevent clogged ducts and mastitis.
- Relieve breast tension and engorgement.
- Support a smoother, uninterrupted breastfeeding journey.

NIPFIT® FUNCTION:

Correction of flat or inverted nipples.

This function is designed to help correct flat or inverted nipples and support milk extraction through gentle, progressive stimulation:

- It helps the nipple protrude outward.
- It facilitates the baby's latch on to the breast.
- It supports the initiation and continuation of breastfeeding.

IMPORTANT: The NipFit® function only works in single mode.

CONTRAINDICATIONS

Do not use the NipFit® feature if:

1. You have open sores, cracks, lesions, or active infections on your nipple or areola.
2. You have had recent breast surgery.
3. You have a diagnosed condition of nipple inversion caused by disease or trauma, not congenital. If this is the case, consult your doctor first.
4. You experience sharp pain from the first application. Discontinue use immediately.
5. You have particularly sensitive skin or suffer from local dermatological conditions, such as eczema or psoriasis.
6. You are pregnant. Suctioning may stimulate contractions.

**WARNING**

The NipFit® feature is intended for postpartum use only. Do not use during pregnancy.

WARNINGS DURING USE

- Always start at the lowest level, Level 1. Do not increase the intensity until the nipple has gradually adapted, even if the sensation is comfortable.
- Do not exceed 5-10 minutes of continuous use of the NipFit® feature per session, especially in the first few weeks.
- Do not perform more than 2 correction sessions per day. Excessive use may cause irritation, swelling, or micro-tears to the nipple.
- Discontinue use immediately if you experience sharp pain, abnormal redness, persistent swelling, blisters, or discharge of fluid other than milk.
- This device is not a certified medical device for clinical therapy and does not replace consultation with a midwife, physician, or International Board Certified Lactation Consultant (IBCLC).
- A Grade 3, severely adherent inverted nipple requires medical evaluation before using any correction device.
- Prolonged, high-intensity use may cause swelling of the areola, temporarily making it more difficult for the baby to latch on. This can have the opposite effect to the desired one.

INSTRUCTIONS FOR SAFE USE:

- Make sure the silicone cup is the correct size, including the 24 mm flange or 20/22 mm reducers, before activating the NipFit® feature. A cup that is too large can amplify the suction on the areola in an uncontrolled manner.
- Before the session, apply a few drops of breast milk or pure lanolin cream to soften the area.
- During the NipFit® session, the nipple should be centered in the cup. An incorrect position can cause asymmetric suction and irritation.
- If you do not notice any improvement after 4-6 weeks of proper use, consult a healthcare professional.

BREAST PUMP CONTROL UNIT

- | | |
|---|---|
| A1. On and Off Switch button
(long press to turn on) | B1. Level indicator |
| A2. Single/Double mode button | B2. Working time indicator(min) |
| A3. Working level decrease button | B3. Expression mode indicator |
| A4. Working level increase button | B4. Stimulation mode indicator |
| A5. Working mode switch button | B5. Drainage/Anti-stasis mode indicator |
| | B6. Single mode indicator |
| | B7. Minute indicator icon |
| | B8. Battery indicator |
| | B4+B5. NipFit® function activated |

Intensity levels:

- 1 - 9 This is the Stimulation mode with fast and short suction rhythm.







- 1 - 9 This is the Expression mode. This simulates the suction of a baby. The rhythm of this phase is slower, deeper and longer.

- 1 - 9 This is the Drainage / Anti-stasis mode. This helps preventing Galactostasis.


ASSEMBLY OF THE BREAST PUMP

1. Insert the breast cup (Package Content No. 2) in the breast pump as shown.
2. Fit the anti-spill silicone (Package Content No. 4) together with the anti-spill cover (Package Content No. 5) on to the breast pump body (Package Content No.2) and press firmly down as shown in the figure.
3. Fix the suction valve (Package Content No. 6) at the lower end of the breast pump see the figure below.
4. To connect the upper part of the breast pump with the wide neck bottle rotate until it fits close.
5. Insert the connection tube (Package Content No.10) into the interface of the anti-spill upper cover. Insert the other end of the connection tube into the suction slot of the pump. (See picture 6 for suction slot of the pump for SINGLE or DOUBLE pumping mode).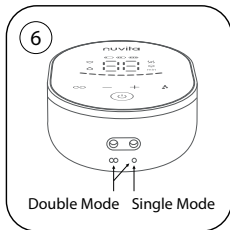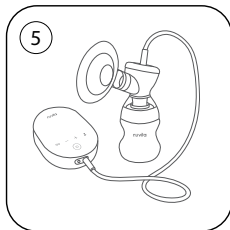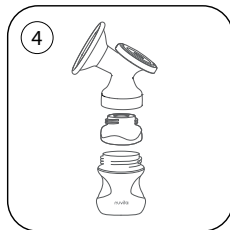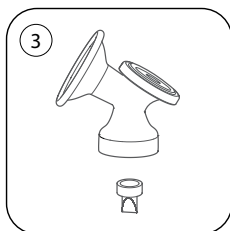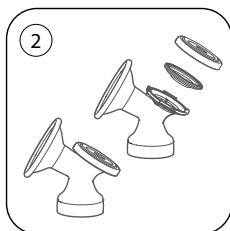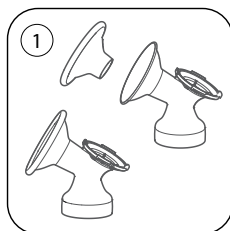

USING THE BREAST PUMP

1. Please clean hands before using the product, ensure the parts of the breast pump are cleaned and assembled accurately as per instruction in this guide.
2. Relax and be at ease while having a normal sitting posture. Use a thermal gel breast pad or a hot towel before starting the pumping process.
3. Use your hand to aim the centre of the breast pump at the nipple and ensure it is tight and lock without air getting in and out.
4. Power on, press A1 power button. The screen of the machine will self test and enter stimulation mode automatically. The factory default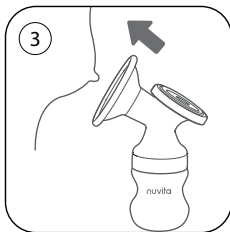

level is level 1. Press A4 to increase the level of suction intensity. After 2 mins of stimulation, the machine will auto switch to expression mode. Press A4 again to increase the level of suction intensity. Machine will auto shut off after 30 mins working.


- Press A5 to enter Expression mode if your milk starts to flow earlier. Press A4 again to increase the level of suction intensity. Machine will auto shut off after 30 mins working.
 - In Expression Mode, Press A5 again to enter stimulation mode again at the same level of suction intensity in previous Stimulation session. After 2 mins of stimulation, the machine will auto switch to expression mode. Machine will auto shut off after 30 mins working.
 - Press A5 to enter Drainage mode when you prefer. Press A4 again to increase the level of suction intensity. Machine will auto shut off after 5 mins.
5. Disassemble, clean and sterilize the non-electronic parts for next use.

INSTALLATION OF AC ADAPTOR

IMPORTANT: Please turn off the machine before connecting with the adaptor.



1. Insert DC round pin of AC power adapter into the DC socket at the Control Unit and insert the AC power adapter into the power socket as shown in the diagram.
2. When battery is low capacity, only the icon  will flicker, indicating that the machine needs to be charged.
3. When charging, the 3 battery icons on the screen will flicker.
4. When machine full charged, the icon  will be always on.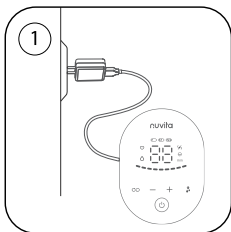


HOW TO CHOOSE THE RIGHT BREAST CUP

The milk extraction should be a pleasant action, and the size of the breast cup greatly affects the type of experience that is lived while using the breast pump. The Nuvita interchangeable breast cups, compatible with all models of the Materno Smart X line, are available in 3 sizes: 20, 24 or 28 mm.

1. With a ruler or a measuring tape, measure the diameter of the nipple (in mm), passing through the centre and excluding the areola.
2. Based on the mm measured, choose the breast cup that is the closest to the measured value.

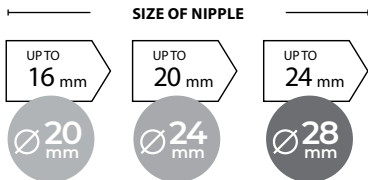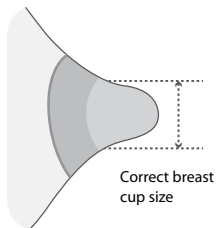
HOW TO UNDERSTAND THE CORRECT BREAST CUP SIZE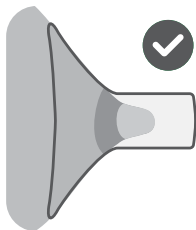

GOOD FIT

If the nipple remains in the centre when operating the breast pump and it moves freely without feeling any discomfort, the breast cup used is in the correct size.

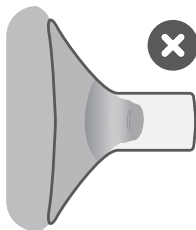TOO SMALL

If the nipple rubs along the side of the tunnel, becomes red and gives discomfort, the breast cup is too small.

TOO LARGE

If the nipple and excessive areola are pulled into the tunnel and become whitish, the breast cup is too large.

IMPORTANT SAFETY MEASURES

- Check before each use.
- Discard at the first sign of damage or wear.
- If you experience any problems or pain, consult your doctor.

ASSEMBLY AND USE

- Connect the breast cup to the connector of the Nuvita Materno Smart breast pump.
- Verify that the breast cup and connector are securely assembled.

CLEANING

- Clean the breast cup immediately after use to avoid drying of milk residues and to prevent the formation of bacteria.
- For cleaning, use only drinking water.
- If you use a brush, be careful not to damage the components.
- Clean the breast cup in a hot water for 2 minutes.
- Recommend cold sterilisation.

AFTER EACH USE

- Rinse the breast cup with cold clean water.
- Clean with plenty of warm water and dish washing detergent.
- Rinse with clean cold water.
- Dry with a clean cloth or let it dry on a clean cloth.
- Store in a clean storage bag, cloth or paper until next use.

BEFORE EACH USE

- Thoroughly wash your hands with soap and clean water before touching your breasts.
- Clean the breast and surrounding area with a damp cloth.
- Do not use soap or alcohol, as they can dry out the skin and cause cracked nipples.

WARRANTY - TERMS AND CONDITIONS

This product is covered by a 24-month warranty for manufacturing defects, starting from the date of purchase proven by receipt or invoice. In case of purchase with invoice, the legal warranty is reduced to 12 months. The 24-month warranty **does not include** damage resulting from normal wear and tear of the product, aesthetic damage, accessories or consumables (for example batteries, heads or components subject to wear). **Batteries are guaranteed for a period of 6 months.**

INVALID WARRANTY: The 12/24-month legal warranty lapses in the following cases:

1. The product has been damaged due to improper use not in accordance with the instructions in the manual.
2. The product has been modified, tampered with or opened.
3. The malfunction is caused by insufficient maintenance of the components, accessories or supplies (e.g. oxidation, immersion of electrical parts in liquids, residues blocking the sensor, leakage of corrosive liquid from the batteries).

WARRANTY EXCLUSIONS: The following are excluded from the 24-month legal warranty:

1. The costs for the replacement or repair of parts subject to wear and those relating to ordinary maintenance of the product.
2. The costs and risks associated with transporting the product to and from the point of sale or the authorised centre for warranty management.
3. Damage resulting from incorrect installation or improper use not in accordance with the instructions in the manual.
4. Damage caused by natural disasters, falls, accidental impacts or environmental conditions incompatible with the product.
5. Defects that have a negligible impact on product performance.

LIMITATIONS OF LIABILITY:

The manufacturer, distributor and all parties involved in the sale of the product are not responsible for economic losses or damages resulting from the malfunction of the product. In accordance with the current legislation, they are not liable for direct, indirect, loss of income, savings or other consequences resulting from the use or inability to use the product and/or associated documentation (paper or electronic), including the lack of service.

For further details on the assistance service, please consult the Conditions of Sale available on the website **www.nuvitababy.com**.

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All rights reserved
Anteprima Brands International Ltd