nuvitamed

IT Aerosol MESH
indossabile

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EN Wearable MESH
nebuliser

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DE Tragbarer MESH-
Vernebler

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EN Instruction manual

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
Thank you for purchasing this Nuvita 5078 Wearable MESH Nebuliser. Please read the Operating Manual carefully before using the product and put it in a safe and secure place for reference.





GENERAL INFORMATION

The device is designed for the treatment of respiratory diseases and the prevention of respiratory tract infections.

COMPONENTS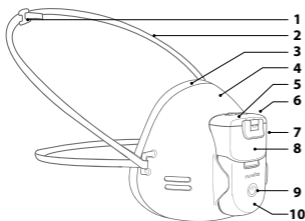

1. Adjustable buckle
2. Elastic headband
3. Silicone edge
4. Adult face mask
5. Medicine container lid
6. Medicine lid closure
7. Medicine container release button
8. Medicine container
9. Switch button
10. Operation LED light
11. Medicine container sealing ring
12. Mouthpiece
13. Children mask
14. Adult mask
15. Silicone support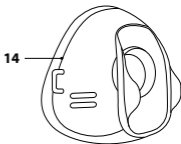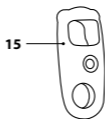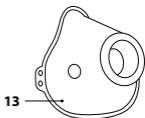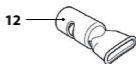

USAGE

The nebuliser is designed for respiratory diseases treatment and respiratory tract infection prevention. It is widely used in curing diseases of respiratory system, such as cold, fever, bronchitis.

APPLICABILITY


The nebulizer can be used by adults, the elderly, infants and children under the supervision of an adult. The nebuliser can be used at home-hold environment, and it complies with requirement of IEC 60601-1-11.

Contraindications: 1) The device cannot be applied to anaesthesia breathing system and respiratory system. 2) Infants and children must under adults' supervision. 3) Pentamidine is not applicable to the device.

WORKING PRINCIPLE

The nebulisation occurs through the high-speed vibration of the mesh plate which vaporizes the medicinal liquid that comes out through the micro mesh holes into very small particles.

1. Atomized particles
2. Piezoelectric ceramic plate
3. Mesh plate
4. Liquid medicine
5. Mesh micropore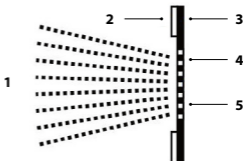

MAIN PERFORMANCE INDEX

Please see Table, it was tested by 0.9% physiological saline under the condition of temperature $25 \pm 3^{\circ}\text{C}$, humidity $60\% \pm 5\% \text{R.H.}$



MAIN PERFORMANCE INDEX	
Working frequency	1 to 3 MHz
Nebulisation rate	$\geq 0.2 \text{ ml/min}$
Medicinal liquid temperature	20°C to 40°C
Noise level	$\leq 25 \text{ dB}$
Particle size	$5 \mu\text{m} \pm 25\%$

OPERATION

To prevent the spread of infections, single person use is recommended. Before using the nebuliser for the first time and after each use, clean it and disinfect every part following the guides in this manual. Dry it well before storing it.

CONNECT THE POWER SUPPLY

The aerosol is equipped with a built-in lithium battery. Charge it fully before using it for the first time. This appliance can also be used connected to a power supply (not included). The indicator light is: red when the charge level is low and while charging, blue when the device is in use, red and blue when it is charging and in use at the same time, turns off when the device is turned off. After charging, unplug the cable. To charge the device, use only the supplied USB cable.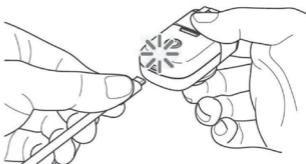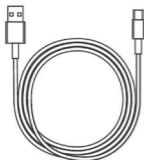

NOTE:

Only use chargers with the following electrical parameters:

1. Input voltage: AC 100~240V,50/60 Hz.
2. Output voltage: DC 3~5V,1~2A.

HOW TO ADD MEDICINE LIQUID

Method A: Add medicine to the container (the medicine cannot be more than 10ml)

1. Open the lid of the container upwards and pour the medicine in.
2. Close the lid and fasten the medicine container buckle.

NOTE:

1. Before adding medicine to the container, please remove the main unit from the adult mask and operate it.
2. After adding the medicine, please check whether there are bubbles. The bubbles will cause the device to malfunction and stop atomizing. If there are bubbles, shake the device to remove the bubbles.
3. If there is an obvious liquid leak, check whether the container lid buckle is fastened and whether the sealing ring inside the medicine container lid is loose or missing.

HOW TO USE THE NEBULISER

The adult face mask has the concave end of the nasal bridge facing up, and the main body medicine chamber facing up. Pinch both sides of the mask to deform the mask, and at the same time buckle the main unit into the mask from the front to form a whole and confirm that the buckle is in place.

The main unit is used with the Mouthpiece or children's mask for handheld use for children's masks, please install the end with the dented bridge of the nose vertically upwards. Please keep the nozzle with a flat mouth and install it horizontally.

NOTE:


If you need to treat through the throat, use a mouthpiece; if you need treatment through the nasal cavity and throat, please choose a mask. The nozzle is suitable for adults and children over 5 years old.


USING THE DEVICE WITH SMALL CHILDREN

To use the device with small children we recommend using the silicone support for handheld use.

1. Insert the device inside the silicone support;
2. Insert the children's mask onto the device;
3. Carry out the inhalation while holding the support with the device inside in your hand, taking care to always keep it in a vertical position.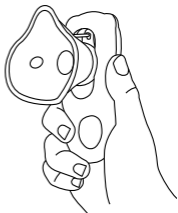

HOW TO WEAR

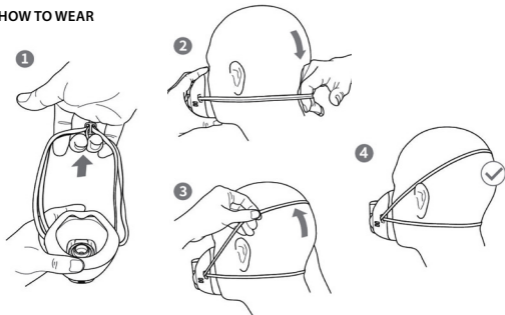1. Hold the mask body in one hand; hold the two headbands in the other hand and pull.
2. Put the mask over the nose and mouth, stretch the two headbands and put them on the back of the neck.
3. Pull the outer one to the top of the head and adjust the position of the two headbands to stable state.
4. If it is too loose or too tight, please adjust the length of the headband and wear it again. If the two headbands are on the upper and lower sides of the ears and are stable, it is the correct wearing state. The sub-adult mask is suitable for children and adults over eight years old.

HOW TO USE THE NEBULISER

Position during treatment

During treatment it is advisable to keep the device in a vertical position or slightly tilted backwards but not tilted sideways otherwise the nebulization may not occur correctly and cause the device to stop. In case of interruption, turn the device back on.


At the end of the nebulization:

1. Turn off the device;
2. Clean the medicine container by removing it from the device using the appropriate release button;
3. Clean the main unit with a damp cloth but without immersing it in any liquid and without using chemical solvents;
4. Carefully dry both the main unit and the medicine container;
5. Install the drug container on the device using the appropriate slide. Scroll until you hear the locking sound.

⚠ CAUTION

1. Device will automatically shut down after 20 minutes of continuous operation.
2. If the red indicator light flashes in the working state, indicating that the battery is low, please charge it.
3. During use, do not shake the device and avoid moving because sudden movements could interrupt operation.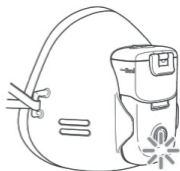

TREATMENT ATTENTION

1. When using the Mouthpiece, let the flat end of the Mouthpiece enter the mouth with the lips slightly closed, and inhale through the mouth.
2. When using the mask, it should fit the face and cover the mouth and nose at the same time.
 - (a). For treatment through the nasal cavity, it is recommended to inhale deeply through the nasal cavity and exhale from the mouth.
 - (b). For treatment through the throat, it is recommended to inhale deeply through the mouth and exhale from the nasal cavity.
 - (c). Also, (a) and (b) can be used alternately during treatment.
3. Use the mouthpiece correctly, avoiding directing the spray onto the tongue. Don't bite the mouthpiece.
4. During nebulization treatment, please keep a slow and deep breathing rhythm and inhale more.
5. During the treatment, the medicinal liquid will condense on the skin. After the treatment is completed, please clean the face in time.
6. Don't swallow the atomized liquid, and keep the atomized liquid on the affected area as much as possible.

INDICATOR STATUS DESCRIPTION		
Indicator status	Mode	Direction
Blue light stays on	Normal	Can be used
Flashing red light	Cleaning	Cleaning function in progress
Flashing red light during use	Low battery	Recharge battery
Red light flashes while charging	Charging	
Red light stays on	Charging is complete	Unplug the charging cable

CLEANING AND DISINFECTING

CLEANING AND DISINFECTION OF MEDICINE CONTAINER

1. Open the lid of the container after each use and pour out the remaining liquid.
2. Pour a small amount of hot water into the container (water temperature 40°C -50°C), close the lid, shake it and empty it;
3. Pour 2ml of hot water again, press and hold the power button for 5 seconds to enter cleaning mode (intermittent misting, red light on). Once the hot water runs out, press the power button to turn off and wipe off the remaining medicine with clean medical gauze.
4. Rinse the inside and outside walls of the container and lid with pure water. After cleaning, use a clean medical gauze to gently wipe away the water. The vibrating mesh can be cleaned with a medical cotton swab and the action should be as gentle as possible.
5. The medicine container can be immersed in 75% medical alcohol for at least 3 minutes for disinfection, then drain it and place it in a clean place to dry.

REMOVE THE MEDICINE CONTAINER

Press the release button down and, at the same time, pull the container backwards to separate it from the main unit.

NOTE

1. During the cleaning and disinfection process it is necessary to use clean and sterilized medical gauze and medical cotton swabs.
2. Do not use paper towels or other cloths to dry the water, otherwise dust or lint of cloth may remain on the parts, preventing the device from working.
3. If the metal mesh plate is blocked (low or no atomization), pour 2 ml of hot water, max 60°C, into the medicine container, press the power button for 5 seconds to enter cleaning mode and atomize until 2 ml of liquid is used up.
4. Turn off the device then open the lid of the medicine container and immerse it in warm water or a solution of water and vinegar for about 3 minutes, then rinse with pure water.
5. Do not touch the mesh with sharp and/or hard objects.
6. Do not use detergents other than those recommended. Turn off the power and unplug the power cord when cleaning and disinfecting.
7. Always open the lid of the medicine container when you immerse it in water or alcohol to clean it.

HOW TO CLEAN THE MAIN UNIT

1. If there are stains on the outer surface of the main unit, moisten a gauze with 75% medical alcohol, wipe it gently, and then dry it with dry gauze.
2. A dry gauze can be used to clean the electrodes on the main unit and the medicine container to ensure that the electrodes are connected, and the device can work normally.

**ATTENTION:**

Do not use other volatile liquids to wipe the main body of the device. For example: benzene or thinner. **The main unit is not waterproof** and cannot be washed or immersed in any liquid.

3. After cleaning and drying, push the medicine container forward from the back of the main unit, then push it until it locks onto the main unit.

CLEANING AND DISINFECTION OF MOUTHPIECE AND MASK

1. Remove the mouthpiece or child mask and adult mask.
2. Rinse with pure water for 2~3 minutes.
3. Use 75% medical alcohol to clean all surfaces of the mask and mouthpiece for more than 3 minutes for disinfection.
4. Masks and mouthpiece can be heat sterilized in boiling water.
5. Dry the water with a medical gauze.
6. The soft rubber ring of the mask can be disassembled and reinstalled after cleaning.

**STORAGE AND MAINTENANCE**

STORAGE CONDITIONS:

1. Temperature: - 20°C ~ 55°C;
2. Relative Humidity: ≤ 80% R.H.
3. Air Pressure : 70.0 ~ 106.0 kPa;
4. Other requirements: No direct sunlight, Ventilation, No corrosive gas, No heating device, or Open fire.

STORAGE INSTRUCTIONS:

Clean and disinfect the device after each use. Keep the device assembled and store it in its case or in a clean environment.

MAINTENANCE INSTRUCTIONS

1. Use the device in normal working conditions and for personal use only;
2. Do not operate the device near the heating device or open fire. Do not dry the device and its accessories with the microwave, oven drying, hair dryer, etc.
3. Prevent the nebuliser and its accessories from coming into contact with corrosive liquids or gases.
4. Do not install the main unit near power lines.

SAFETY INFORMATION AND CARE

SAFETY INFORMATION

1. Read all information in the instruction manual and any other documentation included in the package before using the unit.
2. Do not subject the device or any of its components to strong shocks, such as dropping them on the floor.
3. Do not use the device or touch the charger plug or power switch with wet hands.
4. Do not immerse the main unit in water or other liquid, avoid splashing water or other liquid on the main unit or the power supply;
5. Turn it off and dry immediately with a clean cloth.
6. To avoid damage to the device, do not leave the medicine container full of medicine or liquids.
7. This device is approved for human use only.

CARE FOR THE DEVICE

1. The device is a personal medical device, promiscuous use is not recommended.
2. Do not touch the net with your hands or any other object.
3. Keep the device out of reach of infants and unsupervised children.
4. It is recommended to keep the device away from other electrical products with power consumption greater than 2W at least 3.3 meters, to avoid electromagnetic interference; Consult your doctor for users who wear cardiac pacemaker.

 **WARNING:** The device cannot be used for anesthesia!

Guidance and manufacturer's declaration - Electromagnetic immunity

The mesh nebuliser is intended for use in the electromagnetic environment specified below . The customer or the user of mesh nebuliser should assure that it is used in such an environment.

Immunity Test	IEC 60601 Test Level	Compliance level	Electromagnetic environment- guidance
Electrostatic discharge GB/T17626.2	±6kV contact ±8kV air	± 6 kV contact ±2kV, ±4 kV, ±8kV, air	Floor should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%
Electrostatic Transient/ burst GB/T17626.4	Not applicable	Not applicable	Mains power quality should be that of a typical commercial environment
Surge GB/T17626.5	Not applicable	Not applicable	Mains power quality should be that of a typical commercial environment
Voltage dips ,short interruption and voltage variations on power supply input lines GB/T 17626.11	<5%UT, lasting 0.5 cycles (on UT, >95% temporary drop). 40%UT, lasting 5 cycles (on UT, 60% temporary drop). 70%UT, lasting 25 cycles (on UT, 30% temporary drop). ,5%UT, lasting 5S (on UT, >95% temporary Drop)	Not applicable	Mains power quality should be that of a typical commercial or environment. If the user of the mesh nebuliser requires continued operation during power mains interruptions, it is recommended that the mesh nebuliser be powered from an uninterruptible power supply or a battery.
Power frequency (40/60Hz) magnetic field GB/T 17626.8	3 A/m	3 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.

NOTE UT is the a.c. mains voltage prior to application of the test level

Guidance and manufacturer's declaration - Electromagnetic immunity

The mesh nebuliser is intended for use in the electromagnetic environment specified below . The customer or the user of mesh nebuliser should assure that it is used in such an environment.

Immunity Test	IEC 60601 Test Level	Compliance level	Electromagnetic environment- guidance
Conducted RF GB/T 17626.6	3V 150kHz to 80MHz	3V	Portable and mobile RF communications equipment should be used no closer to any part of the device, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommended Separation Distance $d = 1.2 \sqrt{P}$ $d = 1.2 \sqrt{P}$ 80MHz ~ 800MHz $d = 2.3 \sqrt{P}$ 800MHz ~ 2.5GHz Where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in meters (m). Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, should be less than the compliance level in each frequency range b Interference may occur in the vicinity of equipment marked with the following symbol: 
Radiated RF GB/T 17626.3	3 V/m 80MHz ~ 2.5GHz	3 V/m	

Note 1: At 80MHz and 800MHz, the higher frequency range applies.

Note 2: These guidelines may not apply in all situations. Electromagnetic is affected by absorption and reflection from structures, objects and people.

a. Stationary transmitters, such as base stations for wireless (peak/cordless) telephones and terrestrial mobile radios, amateur radio, AM and FM radio broadcasting, and television broadcasting, etc., whose field strength cannot be accurately predicted in theory. In order to assess the electromagnetic environment of fixed radio frequency transmitters, electromagnetic field surveys should be considered. If the measured field strength of the microgrid device is higher than the above radio frequency compliance level, the microgrid device should be observed to verify its normal operation. If abnormal performance is observed, supplementary measures may be necessary, such as re-adjusting the direction or position of the micro-mesh device.

b. During the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m.

Recommended separation distance between Portable and mobile RF communications equipment and the Nebuliser

The Mesh Nebuliser is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the Mesh Nebuliser can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the Mesh Nebuliser as recommended below, according to the maximum output power of the communication equipment.

Rated maximum output of transmitter	Separation distance according to frequency of transmitter/m		
	150KH ~ 80MHz $d = 1.2 \sqrt{P}$	80MHz ~ 800MHz $d = 1.2 \sqrt{P}$	800MHz ~ 2.5GHz $d = 2.3 \sqrt{P}$

0.01	0.12	0.12	0.23
0.1	0.38	0.38	0.73
1	1.2	1.2	2.3
10	3.8	3.8	7.3
100	12	12	23

For transmitters rated at a maximum output power not listed above the recommended separation distance d in meters(m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (w) according to transmitter manufacturer.

Note 1: At 80Mhz and 800MHz, the separation distance for the higher frequency range applies.

Note 2: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

PRODUCT SPECIFICATION

Name	Wearable Mesh Nebuliser
Model	5078 (YS32)
Dimensions	40mm(L) × 61mm(W) × 121mm(H)
Power Source	Lithium battery voltage 3.7V±20%, AC adapter (Input: 10~240V,50/60 Hz,Output: 3V~5V DC, 1~2A)
Battery capacity	400mAh
Mesh plate material	316LVM stainless steel
Safety grade	BF
Power consumption	1.5W+15%
Vibration frequency	120KHz±10%
Nebulization ate	≥0.2ml/min
Noise level	≤ 25dB
Particle size	5µm±25%
Medicine capacity	8ml maximum, 0.5ml minimum
Battery life	60 minutes after fully charged

Operating temperature	Environmental temp: 10°C ~ 40°C, Relative humidity : ≤80% RH, not in Coagulation situation Air Pressure: (86.0 ~ 106.0) kPa
Storage temperature	Environmental temperature: -20°C ~ 55°C Relative Humidity: ≤80% R.H. Air Pressure: 70.0 ~ 106.0 kPa

WARRANTY - TERMS AND CONDITIONS

This product benefits of a warranty of 24 months on material and manufacturing defects, starting from the date of purchase (see sales slip). The warranty of 24 months does not include damages caused by the usual use of parties identified as "consumable" (e.g., batteries, brush heads or parts subject to usury). The legal guarantee of 24 months is void if: **1.** The product has undergone aesthetic damage due to improper use not in accordance with instructions in the manual. **2.** This product has been modified and/or tampered with. **3.** The cause of the failure was due to poor maintenance of the individual components and/or accessories and/or supplies (e.g. oxidation and/or scaling due to the retention of water or other liquids, sediment blocking the sensor, leak of corrosive liquid from batteries). The following is excluded from the legal guarantee of 24 months: **1.** Costs related to replacements and/or repair of parts subject to wear or costs for ordinary maintenance of the product. **2.** The costs and risks involved in transporting the product to and from the store where you purchased or otherwise authorized collection centre to receive the products under warranty. **3.** Damage caused by or resulting from improper installation or improper use not in accordance with the directions in the instruction manual. **4.** Damage due to natural disasters, accidental events or adverse conditions not compatible with the product. **5.** Defects that have a negligible effect on product performance. The manufacturer, distributor and all the parties involved in the sale do not assume any liability for losses and economic damage from any malfunction of the product. In accordance with current regulations the manufacturer, distributor and all the parties involved in the sale are not responding in any case for damages, including direct, indirect ones, loss of net income, loss of savings and additional damage and other details consequences going beyond the damage caused by the breach of warranty, contract, strict liability, wrongdoing or due to other causes, resulting from the use or inability to use the product and/or paper and electronic documents, including the lack of service. For further information on the help service visit the website www.nuvitababy.com

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All rights reserved
SUNGO Cert GmbH
Changzhou Zhengyuan Medical Technology Co., Ltd.
Anteprima Brands International Ltd.