# nuvita

## MATERNO GO

- IT** Tiralatte elettrico  
**ES** Sacaleche eléctrico  
**EN** Electric breast pump  
**FR** Tire-Lait électrique  
**DE** Elektrische Milchpumpe  
**PT** Bomba tira-leite elétrica  
**GR** Ηλεκτρικό θήλαστρο

- IT** Manuale di istruzioni  
**ES** Manual de instrucciones de uso  
**EN** Instruction manual  
**FR** Manuel d'instructions d'utilisation  
**DE** Bedienungsanleitung  
**PT** Manual de instruções  
**GR** Εγχειρίδιο οδηγιών

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Dear Customer, you bought one of the most reliable breast pumps available in the market. We studied this device to be as much user friendly as possible but, we are afraid, you still must spend a small amount of time in reading these instructions to get 100% of the benefits this product can bring you. Keep this manual for future reference.


## PRECAUTIONS

- Please read the instruction manual carefully before using the product.
- This product must be used by adult only with complete attention during the entire process.
- Do not use the power adapter other than the provided.
- Do not warp or knot the connecting tube (Package Content No. 10) during the using process.
- Please put the product or dismantled parts out of baby's reach.
- Do not dismount Breast Pump Control Unit (Package Content No. 9) in any condition; it shall be repaired or replaced by the manufacturer or professional.

## PACKAGE CONTENT

Before assembly and use the electric breast pump to make sure that inside the packaging there are the following accessories, refer to your shop in case of any missing parts:

1. Dust-proof cover
2. Silicone Massage cup
3. Breast Pump Body
4. Anti-spill silicone
5. Anti-spill cover
6. Suction Valve
7. Nuvita Baby Bottle (150ml)
8. Nuvita Bottle Silicone Valve
9. Breast Pump Control Unit
10. Connection Tube
11. Dust proof feeding bottle cover
12. Bottle Teat
13. Feeding bottle ring
14. USB - C Power Adapter
15. Velvet Bag (not shown in the diagram)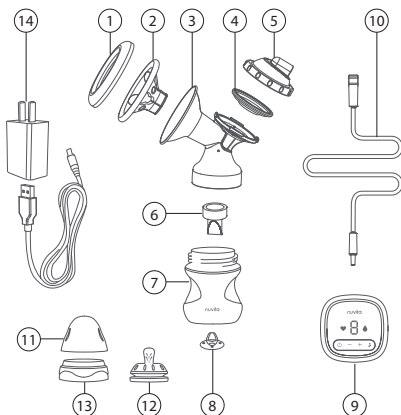

## INSTRUCTION FOR CLEANING

- Before cleaning and sterilization, dismount the parts in contact with the breast or breast milk (apart from Breast Pump Control Unit (Package Content No. 9)).
- Do not clean the Breast Pump Control Unit (Package Content No. 9) in the water.
- Do not sterilize the Breast Pump Control Unit (Package Content No. 9) in the steam sterilizer.
- Do not hot sterilize the connection pipe. Cold sterilization only. Otherwise cleaning with warm soapy water and then rinse with sterile water. (Package Content No. 10).
- Clean the parts immediately after each use to avoid the residual milk and bacteria growth.
- Put the cleaned parts on the dry clean towel at the shady and cool place.


## PRODUCT FEATURES

A rechargeable electric single breast pump model for maximizing milk extraction process. It's designed to simulate the suction rhythm of baby.

1. This Breast pump uses a 2 Phases Technology:
  - 1st Phase with 9 Automatic stimulation levels for 2 minutes operation.
  - 2nd Phase with 9 Selectable levels for your best Expression action
2. This device is provided with an anti-reflux protection system which prevents the breast milk to return to the control unit and damaging it during the milk extraction process.
3. Your 1282C is provided with memory function
4. Your 1282C is portable (AC/DC and Battery operated) so suitable for travelling and perfect for breast pumping activity at work.

## DESCRIPTION OF PUMP CONTROL UNIT

- A. On and Off Switch button
- B. Mode Switch button
- C. Increase Suction Level button
- D. Decrease Suction Level button
- E. Level indicator
- F. Stimulation mode indicator
- G. Expression mode indicator

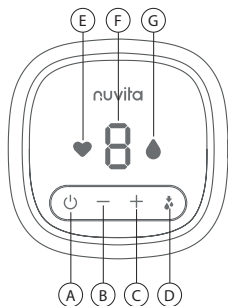


### Frequency levels of absorption of Nuvita 1282C:



♥ 1 to 9 This is the Stimulation mode with fast and short suction rhythm.

💧 1 to 9 This is the Expression mode. This simulates the suction of a baby. The rhythm of this phase is slower, deeper and longer.

## ASSEMBLY OF THE BREAST PUMP

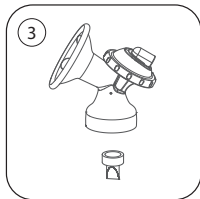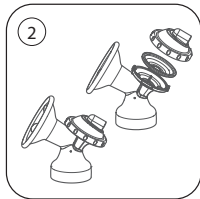
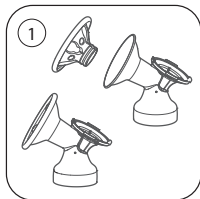
1. Insert the silicone massage cup (Package Content No. 2) in the breast pump as shown.
2. Fit the anti-spill silicon (Package Content No. 4) with the anti-spill cover (Package Content No. 5) as shown below. Aim at the sliding slot and screw the anti-spill cover as shown in the figure.
3. Fix the suction valve (Package Content No. 6) at the lower end of the breast pump see the figure below.
4. To connect the upper part of the breast pump with the wide neck bottle rotate until it fits close.
5. Insert the connection tube (Package Content No.10) into the interface of the anti-spill upper cover. Insert the other end of the connection tube into the suction slot of the pump.
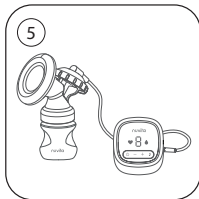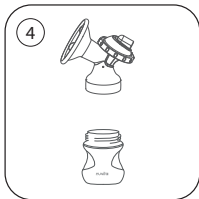
## USING THE BREAST PUMP

1. Please clean hands before using the product, ensure the parts of the breast pump are cleaned and assembled accurately as per instruction in this guide.
2. Relax and be at ease while having a normal sitting posture. Use a thermal gel breast pad or a hot towel before starting the pumping process.
3. Use your hand to aim the centre of the breast pump at the nipple and ensure it is tight and lock without air getting in and out.
4. Power on, long press the power button. The screen of the machine will self-test and flicker. The machine will then enter stimulation mode. The factory default level is level 5. After 2 mins of stimulation, the machine will automatically switch to expression mode. The machine will automatically shut off after 30mins working.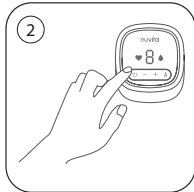
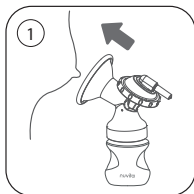


5. Under the Expression mode, press the + button or – button to select any levels from L1 to L9. This device is provided with Memory function. In the next breast feeding it will recall the previous level you selected.
6. Disassemble, clean and sterilize the non electronic parts for next use.

## INSTALLATION OF AC ADAPTOR

**IMPORTANT:** Please turn off the machine before connecting with the adaptor.

1. Insert the USB-C port into the Control Unit and insert the AC power adapter into the power socket as shown in the diagram.
2. When battery is low capacity, the digit on the screen (level indicator) will flicker, indicating that battery needs to be charged.
3. When charging, the line of the digit on the screen will move clockwise and flicker, indicating that the machine is in charging mode as shown in the diagram. When machine full charged, it will auto shut off.

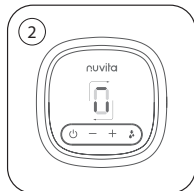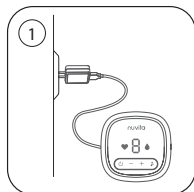## WARRANTY - TERMS AND CONDITIONS

This product has a 24-month warranty on manufacturing defects, starting from the date of purchase (receipt or receipt). The 24-month warranty **does not cover** damage caused by the normal use of accessories or "consumables" (for example, batteries, heads or parts subject to wear ). **Batteries have a 6-month warranty.** The 24-month legal warranty is not valid if: 1. The product has suffered aesthetic damage due to improper use not in accordance with the instructions contained in the manual. 2. This product has been modified and/or tampered with. 3. The cause of the malfunction is due to poor maintenance of the individual components and/or accessories and/or supplies (e.g. oxidation and/or immersion in water or other liquids of the electrical parts, residues that block the sensor, leakage of corrosive liquid from the batteries, ...). The following **are also** excluded from the 24-month legal warranty: 1. Costs relating to the replacement and/or repair of parts subject to wear or costs for ordinary maintenance of the product. 2. Costs and risks associated with transporting the product to and from the store where it was purchased or collection center otherwise authorized to receive the products under warranty. 3. Caused by damage resulting from incorrect installation or improper use or use not in accordance with the instructions in the manual. 4. Damage due to natural disasters, accidental falls or impacts and adverse conditions not compatible with the product. 5. Defects that have a negligible effect on the performance of the product. The manufacturer, distributor and all parties involved in the sale of the product assume no responsibility for economic losses and damages of any malfunction of the product. According to the current legislation, the manufacturer, distributor and all parties involved in the sale are not liable in any case for damages, including defects, direct, indirect, loss of net income, loss of savings and additional damages and other details consequences that go beyond damages caused by breach of warranty, contract, strict liability, tort or other causes, arising from the use or inability to use the product and / or paper and electronic documents, including lack of service. For more information on the assistance service, consult the Conditions of Sale of the site **[www.nuvitababy.com](http://www.nuvitababy.com)**

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All rights reserved
Anteprima Brands International Ltd