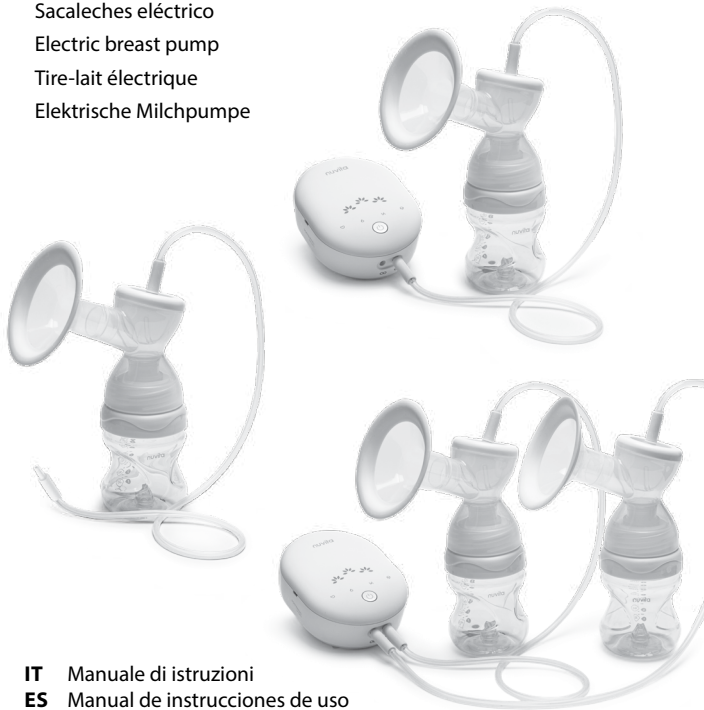
# nuvitamum

## MATERNO *SMART X*

- IT Tiralatte elettrico
- ES Sacaleches eléctrico
- EN Electric breast pump
- FR Tire-lait électrique
- DE Elektrische Milchpumpe

1282X / 1287KDX / 1288X

- IT Manuale di istruzioni
- ES Manual de instrucciones de uso
- EN Instruction manual
- FR Manuel d'instructions d'utilisation
- DE Bedienungsanleitung

# nivitatum

## MATERNO *SMART X*

- PT** Bomba elétrica de mama
- GR** Ηλεκτρικό Θήλαστρο
- NO** Elektrisk brystpumpe
- DA** Elektrisk brystpumpe
- AL** Pompë elektrike e gjirit
- HU** Elektromos mellszívó
- SI** Električna prsna črpalka

1282X / 1287KDX / 1288X

- PT** Manual de instruções
- GR** Εγχειρίδιο οδηγιών
- NO** Bruksanvisningen
- DA** Instruktionsmanual
- AL** Manuali i Udhëzimeve
- HU** Használati útmutató
- SI** Navodila za uporabo

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Dear Customer, you bought one of the most reliable breast pumps available in the market. We studied this device to be as much user friendly as possible but, we are afraid, you still must spend a small amount of time in reading these instructions in order to get 100% of the benefits this product can bring you. Keep this manual for future reference.

## PRECAUTIONS

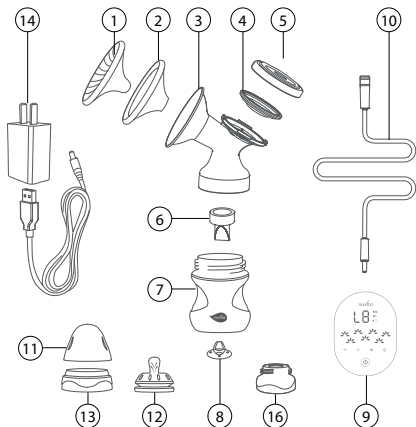
- Please read the instruction manual carefully before using the product.
- This product must be used by adult only with complete attention during the entire process.
- Do not use the power adapter other than the provided.
- Do not warp or knot the connecting tube (Package Content No. 10) during the using process.
- Please put the product or dismantled parts out of baby's reach.
- Do not dismount Breast Pump Control Unit (Package Content No. 9) in any condition; it shall be repaired or replaced by the manufacturer or professional.

## PACKAGE CONTENT

Before assembly and use the electric breast pump to make sure that inside the packaging there are the following accessories, refer to your shop in case of any missing parts:


### 1287 MATERNO SMART X

1. Dust-proof cover
2. Breast cup
3. Breast Pump Body
4. Anti-spill silicone
5. Anti-spill cover
6. Suction Valve
7. Nuvita Baby Bottle (150ml)
8. Nuvita Bottle Silicone Valve
9. Breast Pump Control Unit
10. Connection Tube
11. Dust proof feeding bottle cover
12. Bottle Teat
13. Feeding bottle ring
14. Power Adapter
15. Velvet Bag (not shown in the diagram)
16. Adapter


## 1287KDX MATERNO SMART X KIT DOPPIO

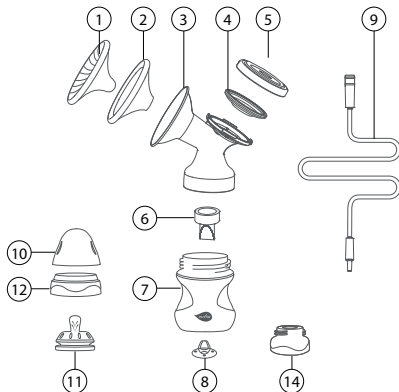1. Dust-proof cover
2. Breast cup
3. Breast Pump Body
4. Anti-spill silicone
5. Anti-spill cover
6. Suction Valve
7. Nuvita Baby Bottle (150ml)
8. Nuvita Bottle Silicone Valve
9. Connection Tube
10. Dust proof feeding bottle cover
11. Bottle Teat
12. Feeding bottle ring
13. Velvet Bag (not shown in the diagram)
14. Adapter

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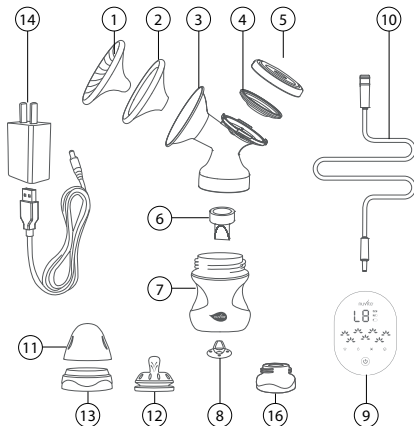1. Dust-proof cover (2x)
2. Breast cup (2x)
3. Breast Pump Body (2x)
4. Anti-spill silicone (2x)
5. Anti-spill cover (2x)
6. Suction Valve (2x)
7. Nuvita Baby Bottle (150ml) (2x)
8. Nuvita Bottle Silicone Valve (2x)
9. Breast Pump Control Unit
10. Connection Tube (2x)
11. Dust proof feeding bottle cover (2x)
12. Bottle Teat (2x)
13. Feeding bottle ring (2x)
14. Power Adapter
15. Velvet Bag (not shown in the diagram)
16. Adapter (2x)

## INSTRUCTION FOR CLEANING

- Before cleaning and sterilization, dismount the parts in contact with the breast or breast milk (apart from Breast Pump Control Unit (Package Content No. 9)).
- Do not clean the Breast Pump Control Unit (Package Content No. 9) in the water.
- Do not sterilize the Breast Pump Control Unit (Package Content No. 9) in the steam sterilizer.
- Do not hot sterilize the connection pipe. Cold sterilization only. Otherwise cleaning with warm soapy water and then rinse with sterile water. (Package Content No. 10).
- Clean the parts immediately after each use to avoid the residual milk and bacteria growth.
- Put the cleaned parts on the dry clean towel at the shady and cool place.

## PRODUCT FEATURES

A rechargeable electric breast pump model for maximizing milk extraction process. It's designed to simulate the suction rhythm of baby.

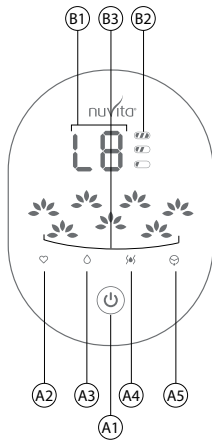
1. This Breast pump uses a 3 Phases Technology:
  - 1st Phase with 9 selectable stimulation levels for 2 minutes operation.
  - 2nd Phase with 9 Selectable levels for your best Expression action.
  - 3rd Phase (selectable on demand) to completely remove of milk from breast to prevent galactostasis.
2. This device is provided with an anti-reflux protection system which prevents the breast milk to return to the control unit and damaging it during the milk extraction process.
3. Your Materno Smart is portable (AC/DC and Rechargeable Battery operated) so suitable for travelling and perfect for breast pumping activity at work.

## PUMP CONTROL UNIT

- A1. On and Off Switch button
- A2. Stimulation Mode button
- A3. Expression Mode button
- A4. Drainage / Anti-stasis Mode button
- A5. Single / Double Mode button (Light ON - SINGLE pump / Light OFF - DOUBLE pump)
- B1. Level indicator
- B2. Battery indicator
- B3. Rhythm indicator







### Intensity levels:

- |   |             |   |
|---|-------------|---|
|  | from 1 to 9 | This is the Stimulation mode with fast and short suction rhythm.  |
|  | from 1 to 9 | This is the Expression mode. This simulates the suction of a baby. The rhythm of this phase is slower, deeper and longer. |
|  | from 1 to 9 | This is the Drainage / Anti-stasis mode. This helps preventing Galactostasis.   |


## ASSEMBLY OF THE BREAST PUMP

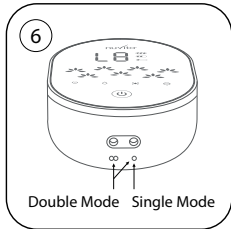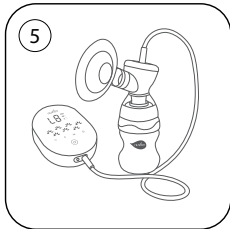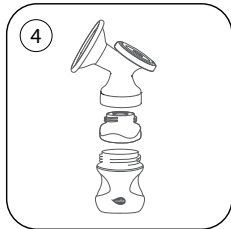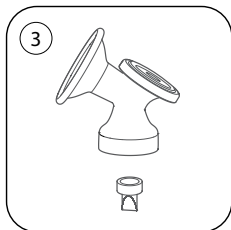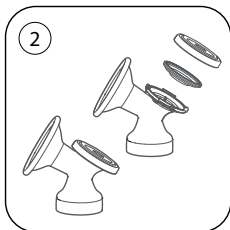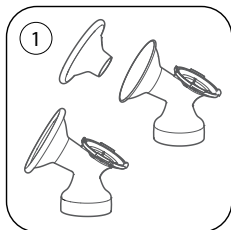1. Insert the breast cup (Package Content No. 2) in the breast pump as shown.
2. Fit the anti-spill silicone (Package Content No. 4) together with the anti-spill cover (Package Content No. 5) on to the breast pump body (Package Content No.2) and press firmly down as shown in the figure.
3. Fix the suction valve (Package Content No. 6) at the lower end of the breast pump see the figure below.
4. To connect the upper part of the breast pump with the wide neck bottle rotate until it fits close.
5. Insert the connection tube (Package Content No.10) into the interface of the anti-spill upper cover. Insert the other end of the connection tube into the suction slot of the pump. (See picture 6 for suction slot of the pump for SINGLE or DOUBLE pumping mode).


## USING THE BREAST PUMP

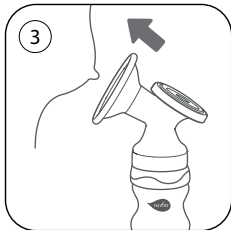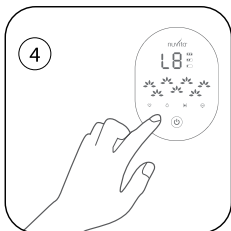1. Please clean hands before using the product, ensure the parts of the breast pump are cleaned and assembled accurately as per instruction in this guide.
2. Relax and be at ease while having a normal sitting posture. Use a thermal gel breast pad or a hot towel before starting the pumping process.
3. Use your hand to aim the centre of the breast pump at the nipple and ensure it is tight and lock without air getting in and out.
4. Power on, press A1 power button. The screen of the machine will self-test and flicker.


- Press A2 will enter stimulation mode. The factory default level is level 1. Press A2 again to increase the level of suction intensity. After 2 mins of stimulation, the machine will auto switch to expression mode. Press A3 again to increase the level of suction intensity. Machine will auto shut off after 30 mins working.
  - Press A3 to enter Expression mode if your milk starts to flow earlier. Press A3 again to increase the level of suction intensity. Machine will auto shut off after 30 mins working.
  - In Expression Mode, Press A2 again to enter stimulation mode again at the same level of suction intensity in previous Stimulation session. After 2 mins of stimulation, the machine will auto switch to expression mode. Machine will auto shut off after 30 mins working.
  - Press A4 to enter Drainage mode when you prefer. Press A4 again to increase the level of suction intensity. Machine will auto shut off after 5 mins.
5. Disassemble, clean and sterilize the non-electronic parts for next use.

## INSTALLATION OF AC ADAPTOR

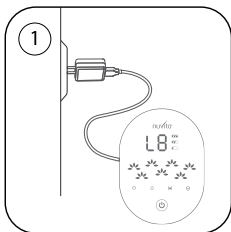**IMPORTANT:** Please turn off the machine before connecting with the adaptor.


1. Insert DC round pin of AC power adapter into the DC socket at the Control Unit and insert the AC power adapter into the power socket as shown in the diagram.
2. When battery is low capacity, only the icon  will flicker, indicating that the machine needs to be charged.
3. When charging, the 3 battery icons on the screen will flicker.
4. When machine full charged, the icon  will be always on.


## HOW TO CHOOSE THE RIGHT BREAST CUP

The milk extraction should be a pleasant action, and the size of the breast cup greatly affects the type of experience that is lived while using the breast pump. The Nuvita interchangeable breast cups, compatible with all models of the Materno Smart X line, are available in 3 sizes: 20, 24 or 28 mm.

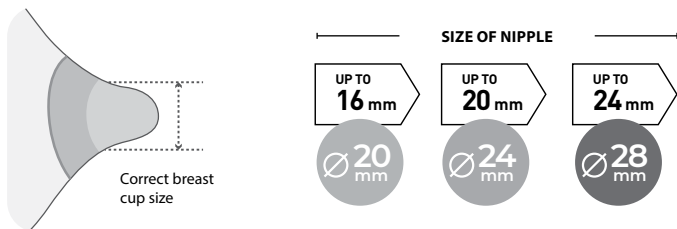1. With a ruler or a measuring tape, measure the diameter of the nipple (in mm), passing through the centre and excluding the areola.
2. Based on the mm measured, choose the breast cup that is the closest to the measured value.


## HOW TO UNDERSTAND THE CORRECT BREAST CUP SIZE

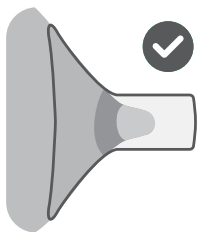
### GOOD FIT

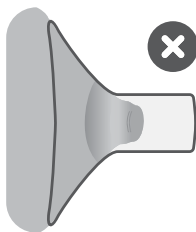If the nipple remains in the centre when operating the breast pump and it moves freely without feeling any discomfort, the breast cup used is in the correct size.

### TOO SMALL

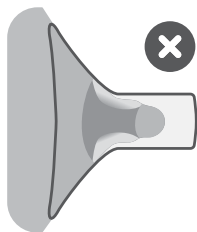If the nipple rubs along the side of the tunnel, becomes red and gives discomfort, the breast cup is too small.

### TOO LARGE

If the nipple and excessive areola are pulled into the tunnel and become whitish, the breast cup is too large.

## IMPORTANT SAFETY MEASURES

- Check before each use.
- Discard at the first sign of damage or wear.
- If you experience any problems or pain, consult your doctor.

## ASSEMBLY AND USE

- Connect the breast cup to the connector of the Nuvita Materno Smart breast pump.
- Verify that the breast cup and connector are securely assembled.

## CLEANING

- Clean the breast cup immediately after use to avoid drying of milk residues and to prevent the formation of bacteria.
- For cleaning, use only drinking water.
- If you use a brush, be careful not to damage the components.
- Clean the breast cup in a hot water for 2 minutes.
- Recommend cold sterilisation.

## AFTER EACH USE

- Rinse the breast cup with cold clean water.
- Clean with plenty of warm water and dish washing detergent.
- Rinse with clean cold water.
- Dry with a clean cloth or let it dry on a clean cloth.
- Store in a clean storage bag, cloth or paper until next use.

## BEFORE EACH USE

- Thoroughly wash your hands with soap and clean water before touching your breasts.
- Clean the breast and surrounding area with a damp cloth.
- Do not use soap or alcohol, as they can dry out the skin and cause cracked nipples.

## WARRANTY - TERMS AND CONDITIONS

This product has a 24-month warranty on manufacturing defects, starting from the date of purchase (receipt or receipt). The 24-month warranty **does not cover** damage caused by the normal use of accessories or "consumables" (for example, batteries, heads or parts subject to wear). **Batteries have a 6-month warranty.** The 24-month legal warranty is not valid if: 1. The product has suffered aesthetic damage due to improper use not in accordance with the instructions contained in the manual. 2. This product has been modified and/or tampered with. 3. The cause of the malfunction is due to poor maintenance of the individual components and/or accessories and/or supplies (e.g. oxidation and/or immersion in water or other liquids of the electrical parts, residues that block the sensor, leakage of corrosive liquid from the batteries, ...). **The following are also excluded from the 24-month legal warranty:** 1. Costs relating to the replacement and/or repair of parts subject to wear or costs for ordinary maintenance of the product. 2. Costs and risks associated with transporting the product to and from the store where it was purchased or collection center otherwise authorized to receive the products under warranty. 3. Caused by damage resulting from incorrect installation or improper use or use not in accordance with the instructions in the manual. 4. Damage due to natural disasters, accidental falls or impacts and adverse conditions not compatible with the product. 5. Defects that have a negligible effect on the performance of the product. The manufacturer, distributor and all parties involved in the sale of the product assume no responsibility for economic losses and damages of any malfunction of the product. According to the current legislation, the manufacturer, distributor and all parties involved in the sale are not liable in any case for damages, including defects, direct, indirect, loss of net income, loss of savings and additional damages and other details consequences that go beyond damages caused by breach of warranty, contract, strict liability, tort or other causes, arising from the use or inability to use the product and / or paper and electronic documents, including lack of service. For more information on the assistance service, consult the Conditions of Sale of the site [www.nuvitababy.com](http://www.nuvitababy.com)

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All rights reserved
Anteprima Brands International Ltd